Handicare 2000 User Manual

Smart seat
Simplicity seat
Style seat

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CONTENT

USER MANUAL	2
OPTIONS	11
IMPORTANT INFORMATION	14
FAULT FINDING	18
ATTACHMENTS	23
<i>A. Reminders</i>	23
<i>B. Emergency operation</i>	24
<i>C. Diagnostic codes</i>	27


FOREWORD

Thank you for choosing a Handicare stairlift. This manual is for the HANDICARE 2000 lift type, and includes instructions for the standard features and optional upgrades.

Make everyday life easier....

ENGLISH	1
NEDERLANDS	31
DEUTSCHE	61
FRANCAIS	91
ESPANOL	121
ITALIANO	151

SMART SEAT

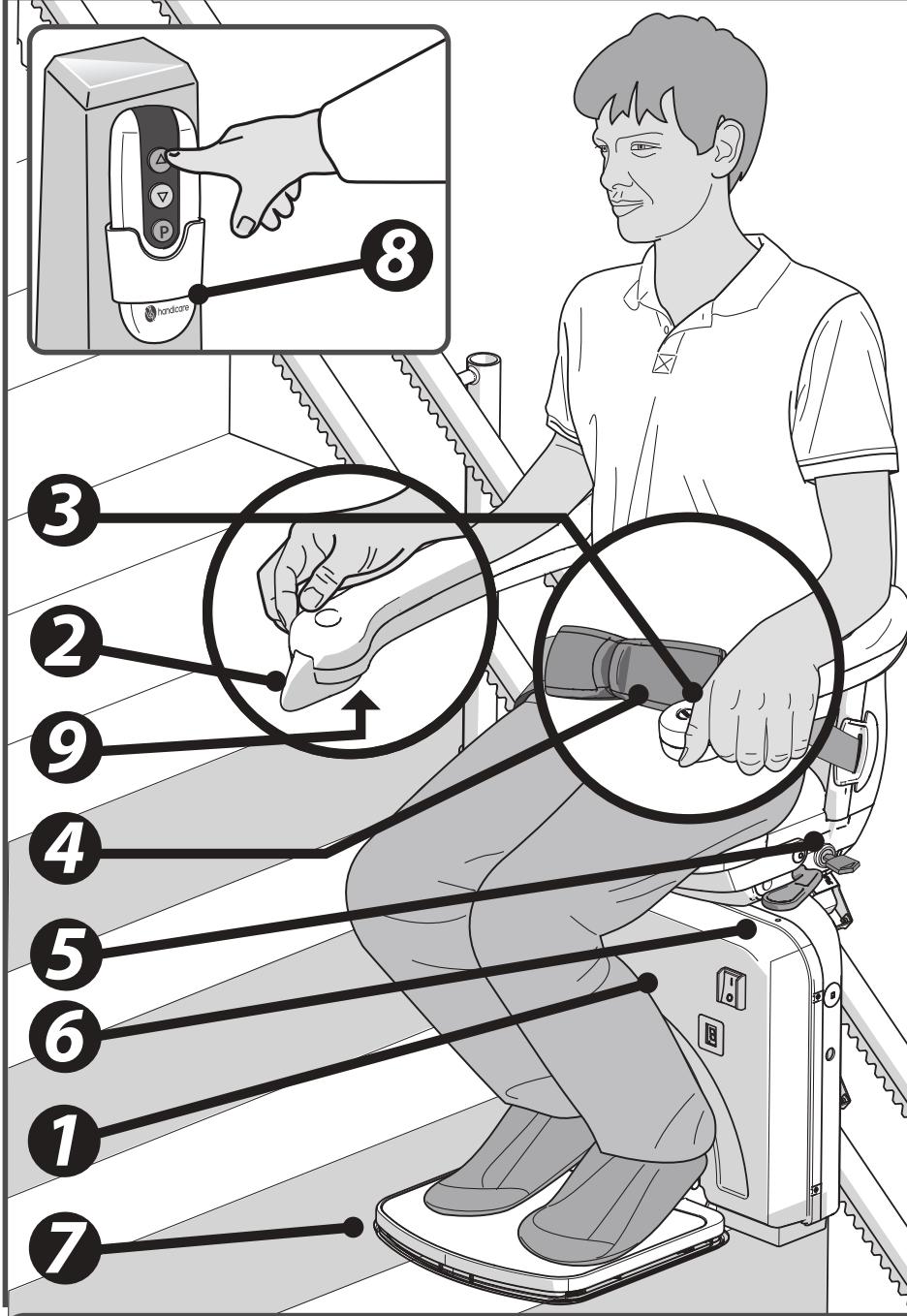
OPTIONS


Automatic Swivel

Powered Footrest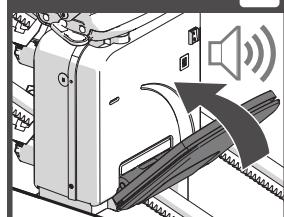

Folding hinge

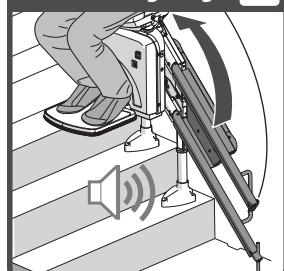The illustration above shows the features used to operate the SMART seat. The SMART seat can be upgraded with the Automatic Swivel seat, Powered Footrest and the Folding hinge, options to create even more seat comfort and user-friendliness.

- [1] Diagnostic Display
- [2] Joystick
- [3] Stop button
- [4] Seat belt
- [5] Key switch
- [6] Seat swivel lever
- [7] Footplate
- [8] Remote control
- [9] Footplate switch


SIMPLICITY SEAT

The illustration above shows the features used to operate the SIMPLICITY seat. The SIMPLICITY seat can be upgraded with the Automatic Swivel seat, Powered Footrest and the Folding hinge, options to create even more seat comfort and user-friendliness.

OPTIONS

Powered Footrest

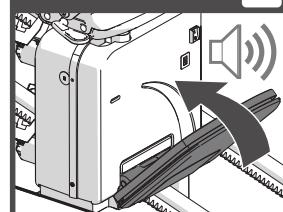Folding hinge


- [1] Diagnostic Display
- [2] Joystick
- [3] Stop button
- [4] Seat belt
- [5] Key switch
- [6] Seat swivel lever
- [7] Footplate
- [8] Remote control
- [9] Footplate switch

STYLE SEAT

The illustration above shows the features used to operate the STYLE seat. The STYLE seat can be upgraded with the Automatic Swivel seat, Powered Footrest and the Folding hinge, options to create even more seat comfort and user-friendliness.


- [1]Diagnostic Display
- [2]Joystick
- [3]Stop button
- [4]Seat belt
- [5]Key switch
- [6]Seat swivel lever
- [7]Footplate
- [8]Remote control
- [9]Footplate switch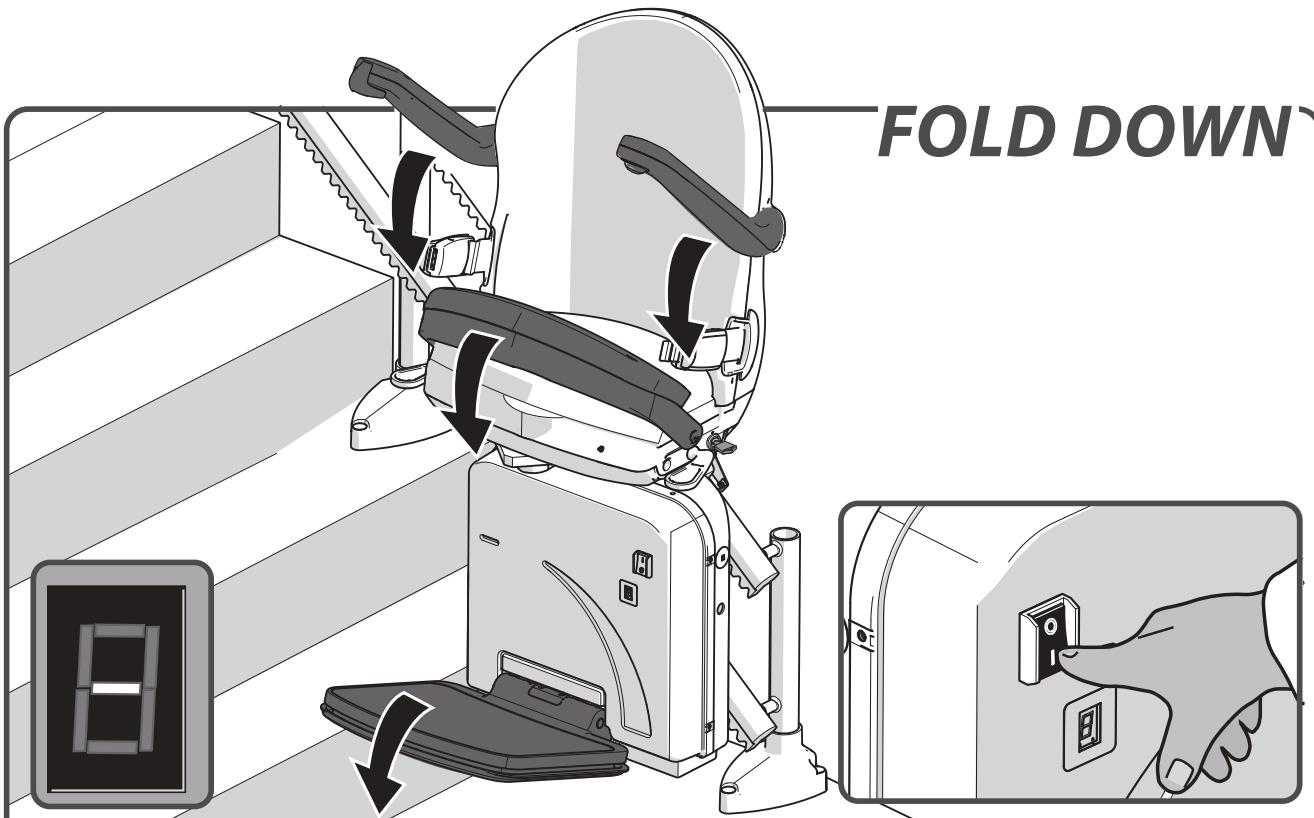



Ensure that the on/off switch is set in the 'ON' position. The diagnostic display will be illuminated when switched on.



Fold down the footplate, the armrests and seat.

[page 12 > OPTIONAL POWERED FOOTREST](#)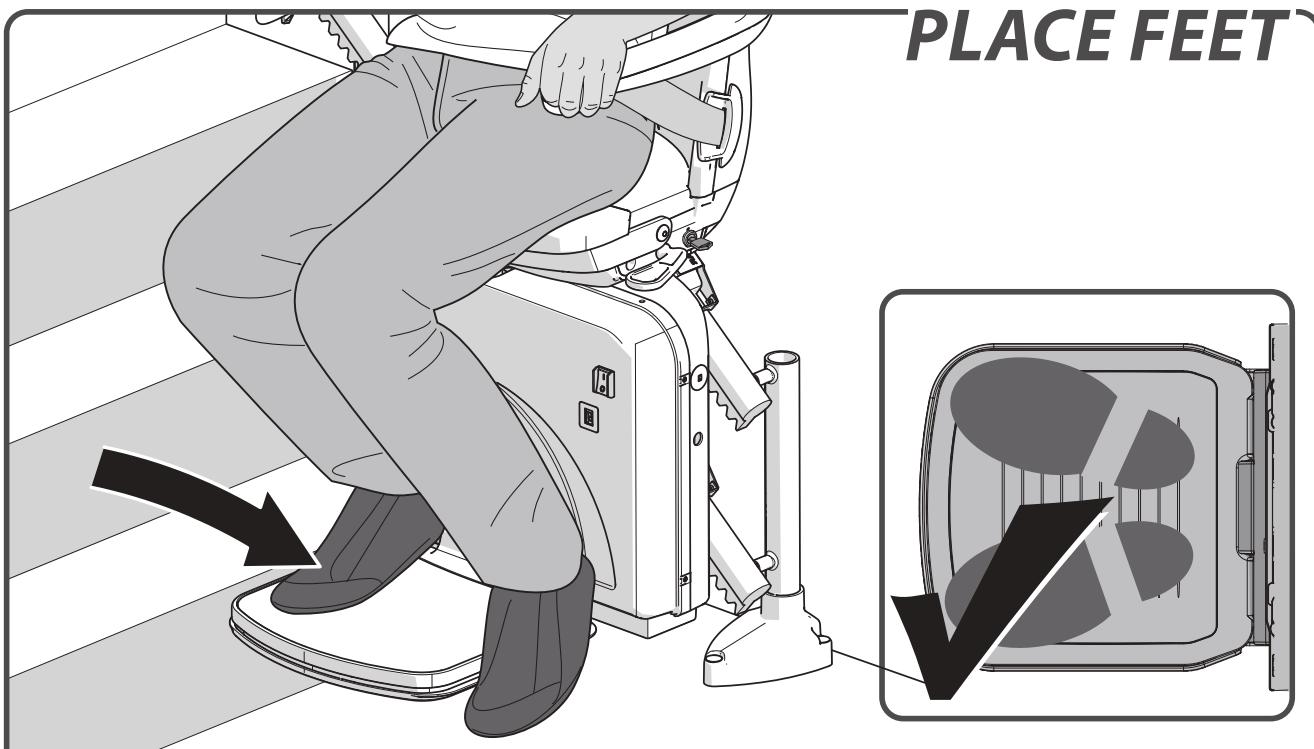

Place your feet on the footplate. Ensure that both feet are completely on the plate.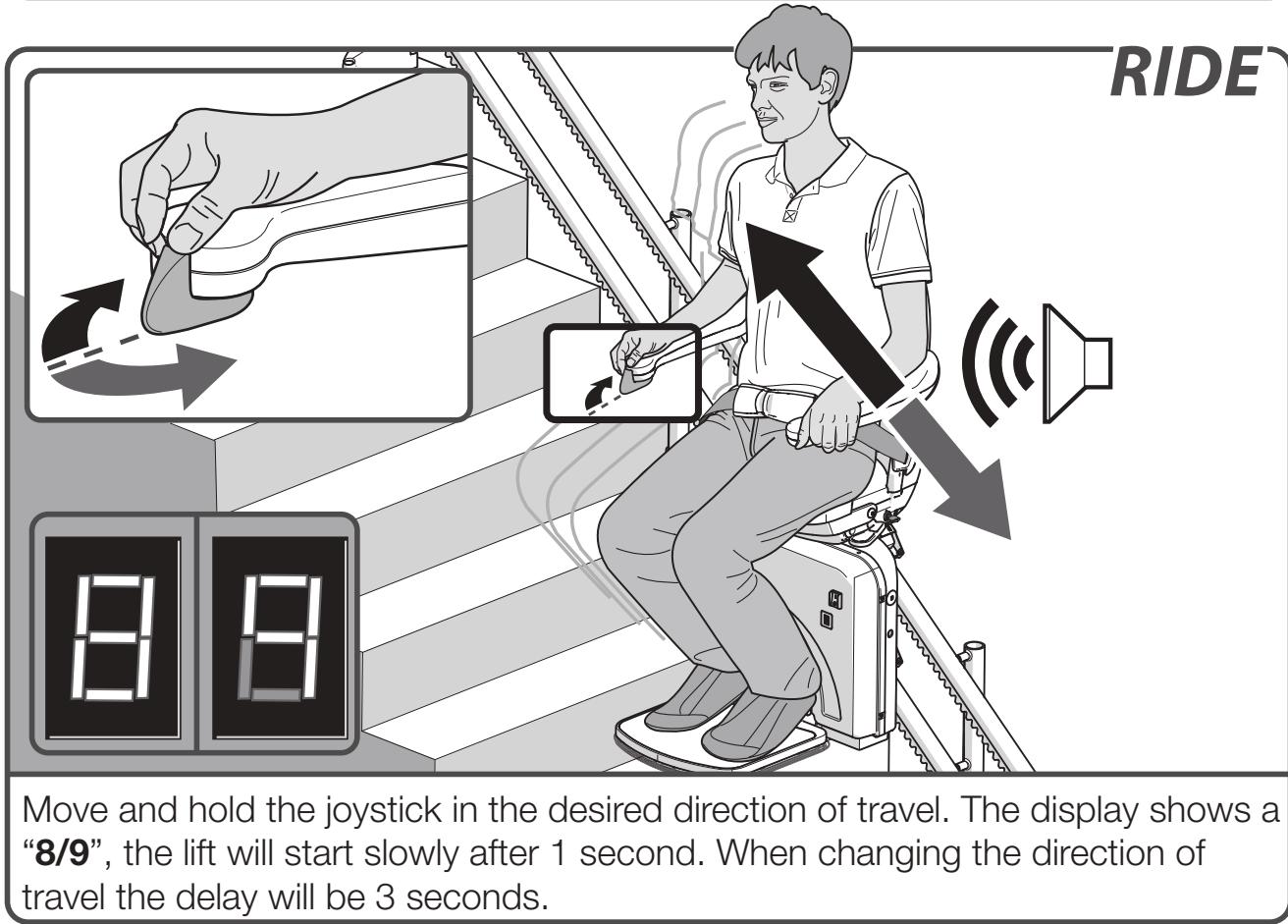
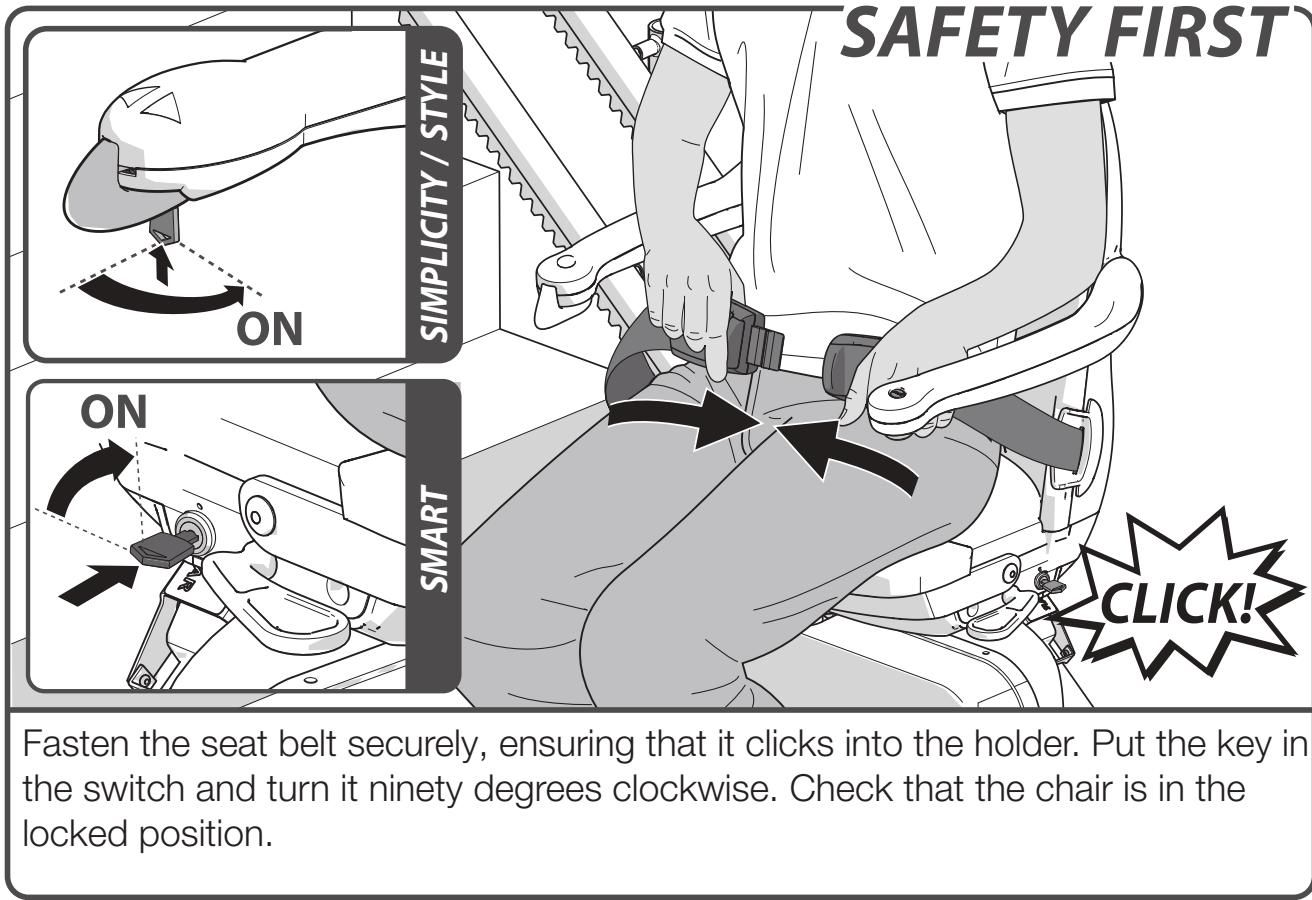


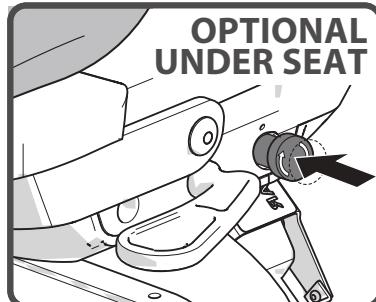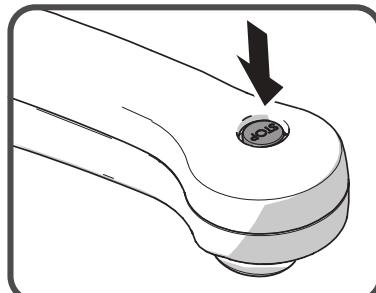STOP**STOP BUTTON**

In an emergency, press the stop button once. The stop button is illuminated. The lift stops straight away.

The stop button can be released by pushing the stop button.



Optional: The stop button can be released by turning the stop button.

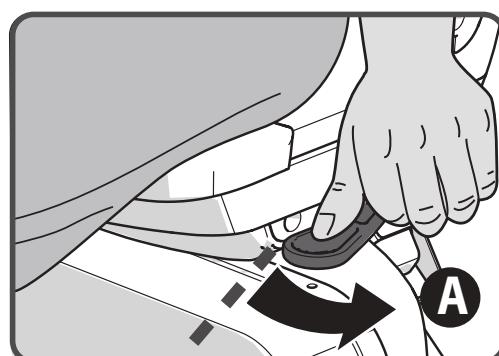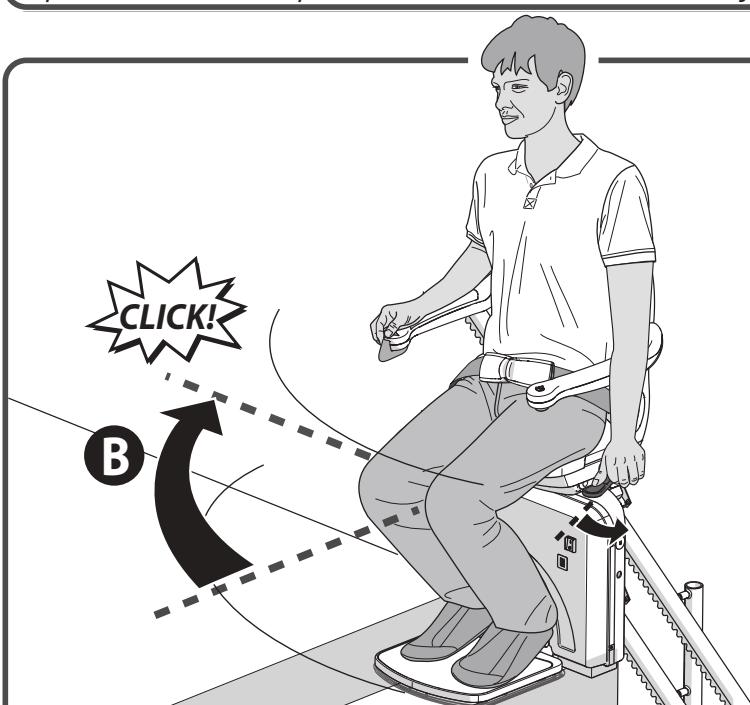
SWIVEL

The lift stops automatically at the end of the rail. You should now swivel the seat manually, making it easier to get on and off. To do this, push down the lever{A}. Swivel the seat until you hear a 'click'. The seat is now in the correct position{B}.


page 11 > OPTIONAL AUTOSWIVEL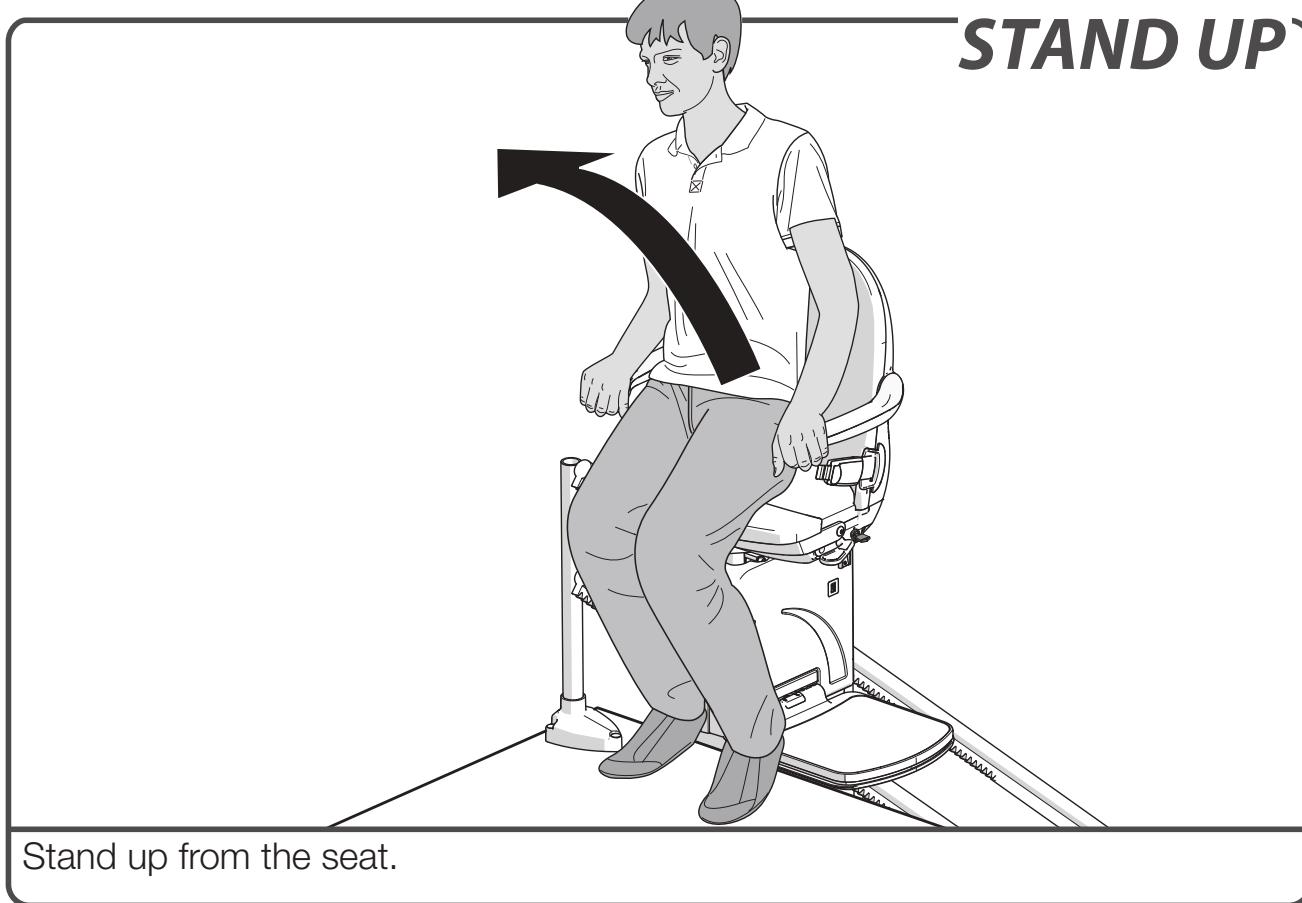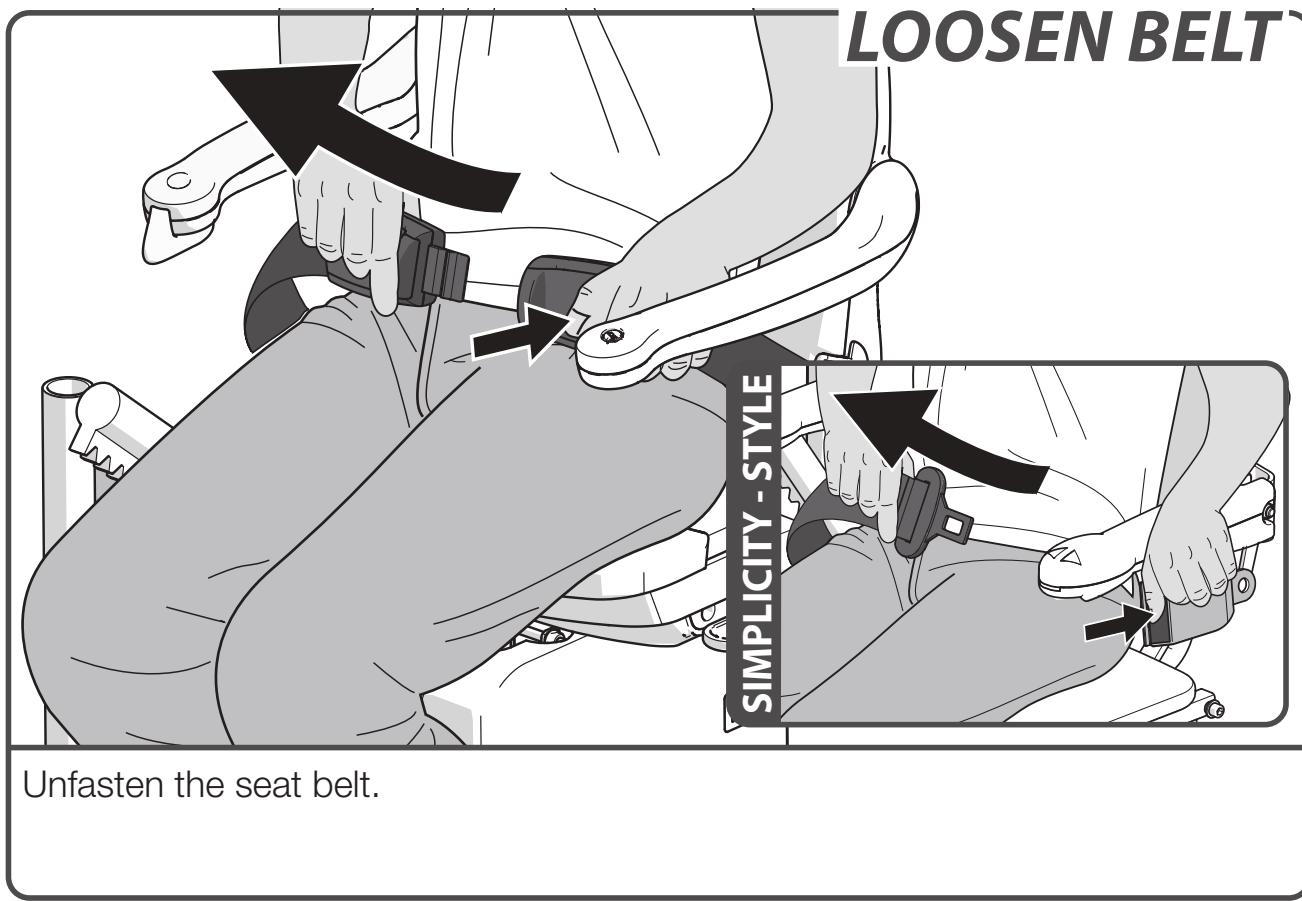

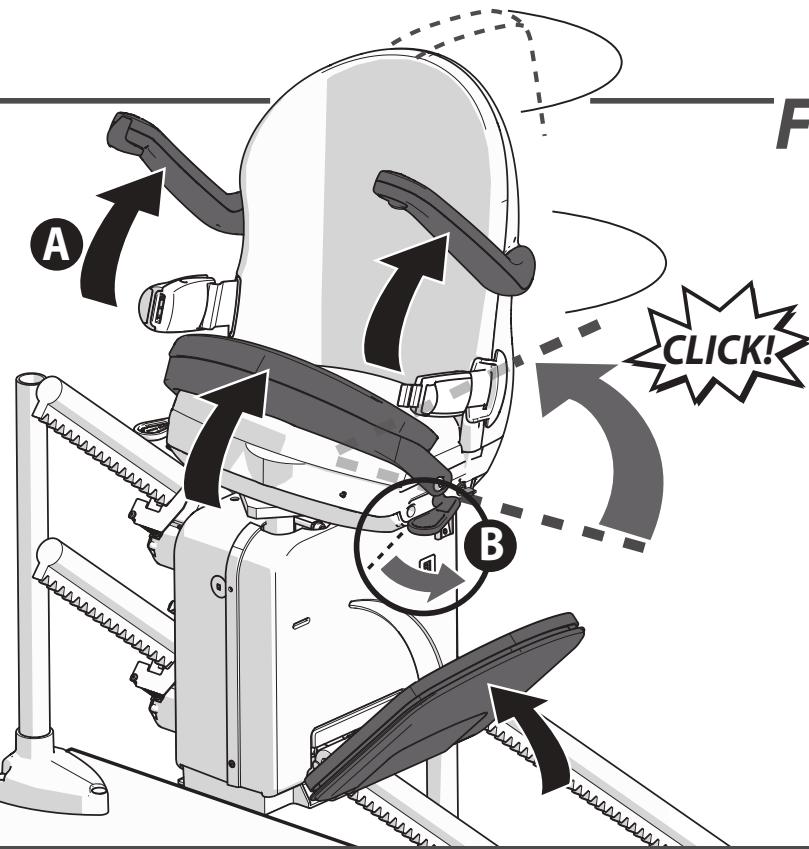
FOLD UP

Fold up the armrests, footplate and seat {A}. Push the handle, located to the left or right under the seat, downwards {B}. Swivel the seat until you hear a 'click'. The seat is now in the locked position.

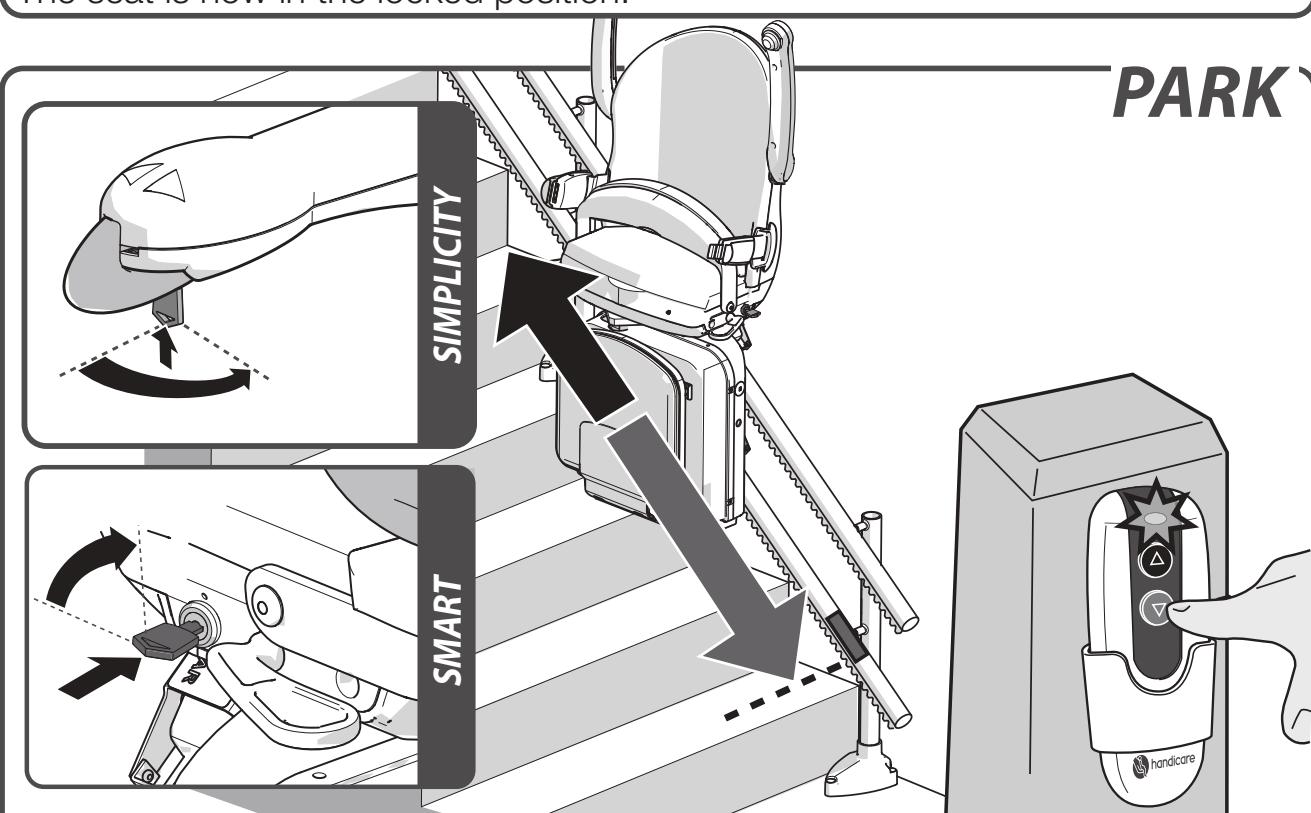PARK


Press and hold the up or down button on the remote control until the lift reaches a charge point. Upon arrival, the lift stops automatically.

CHARGE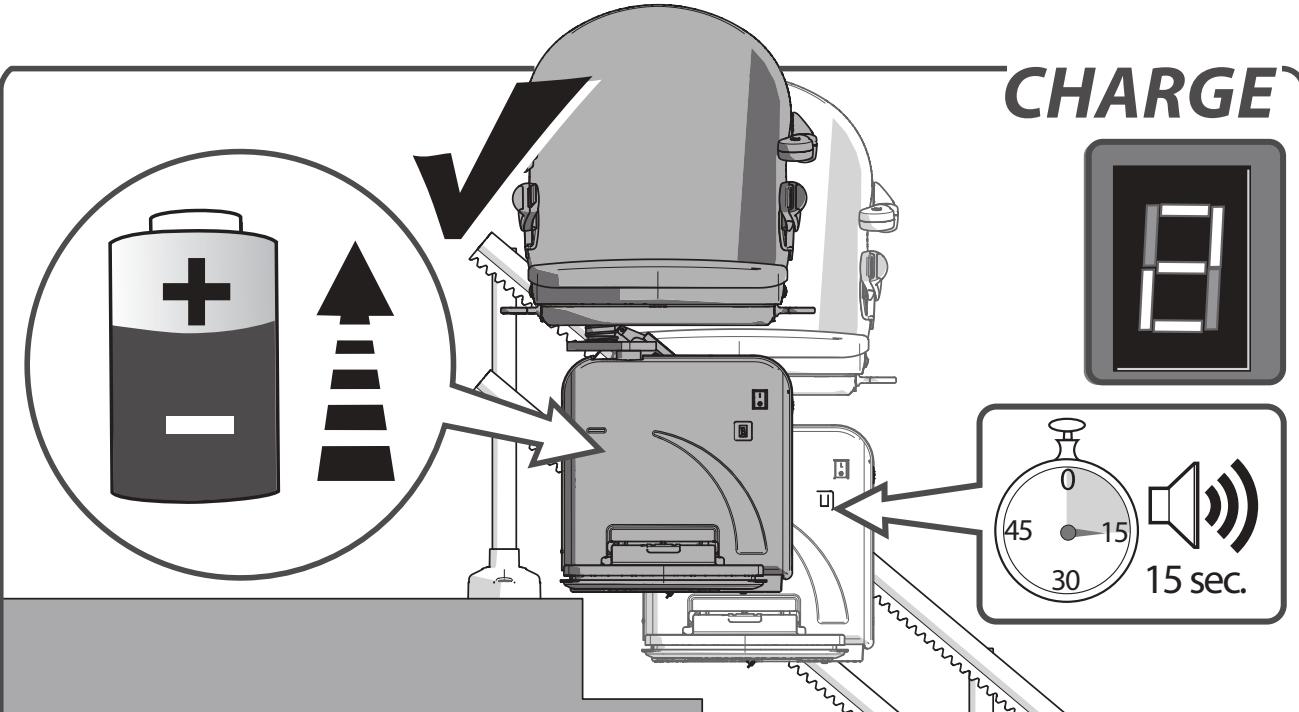

There are charge points for recharging the lift's batteries at the top and bottom of the rail (there may be additional charge points). If the lift is not in the correct position at the charge point, the lift beeps for 15 seconds and the display shows a “**2**”. The lift does not work if the batteries are flat.


CLEANING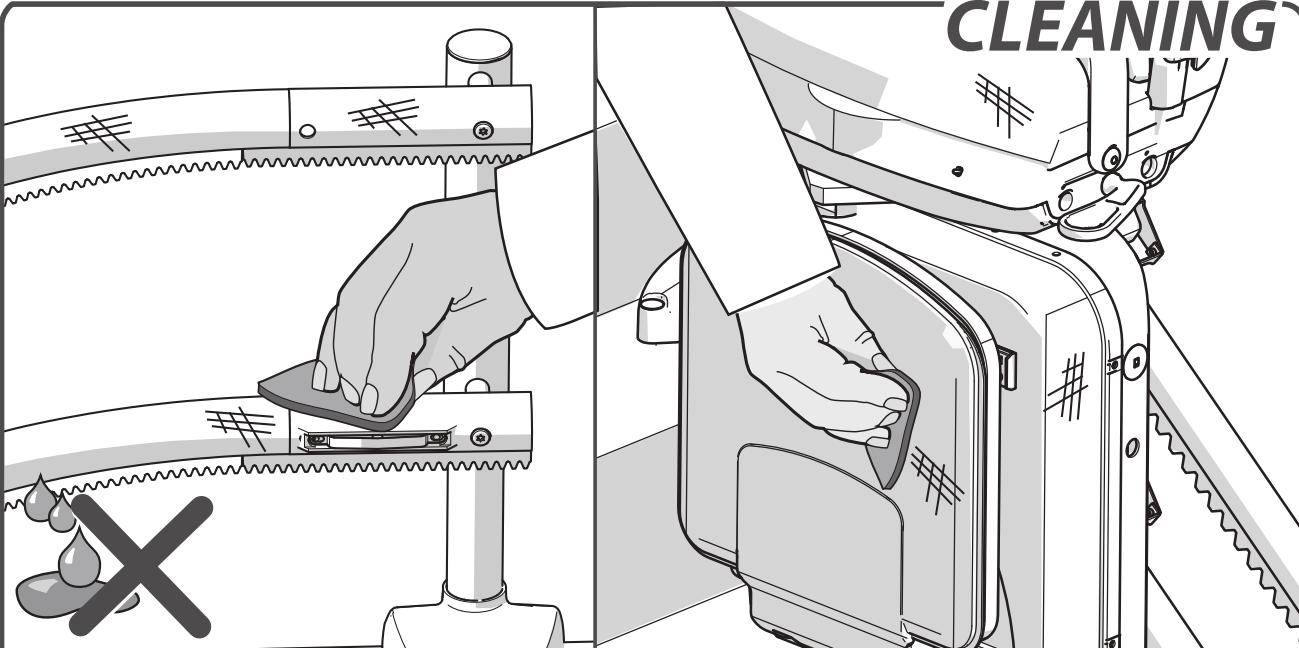

Clean the stairlift regularly with a damp cloth; avoid excessive use of water. Never use an acidic cleaner. Ensure the track is kept free of dirt and dust. Over time the track wheels may leave a mark on the rail.


OPTIONS & FAULTFINDING

AUTOSWIVEL	<i>page 11</i>
POWERED FOOTREST	<i>page 12</i>
HINGED RAIL	<i>page 12</i>
DIAGNOSTIC DISPLAY	<i>page 18</i>
CHAIR POSITION	<i>page 18</i>
STOP BUTTON	<i>page 19</i>
ARMREST	<i>page 20</i>
KEY	<i>page 20</i>
BATTERIES RC	<i>page 21</i>
POWER	<i>page 21</i>
OBSTACLE	<i>page 22</i>
SWITCH	<i>page 22</i>

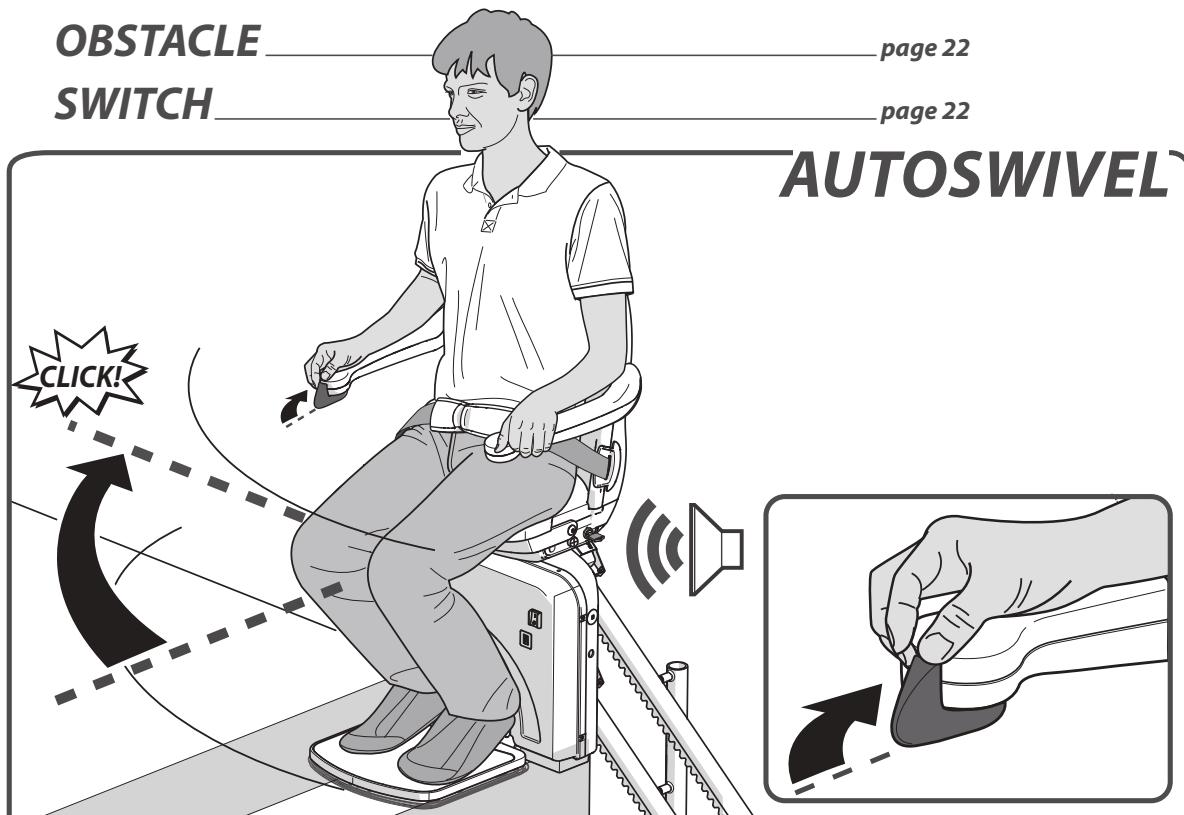
The AUTOMATIC SWIVEL SEAT option let you swivel the seat automatically to face the direction you are going to walk. To do this, push and hold the joystick in the direction of travel.

POWERED FOOTREST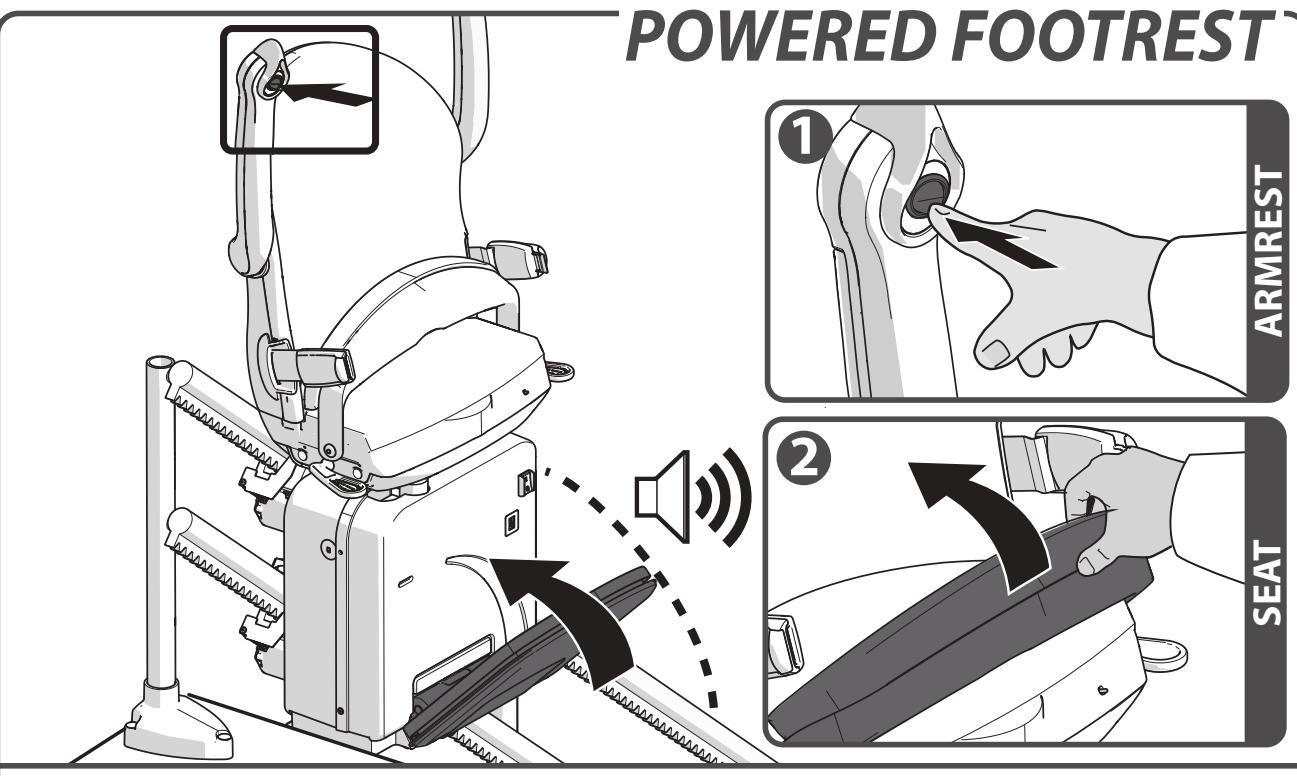


To operate a POWERED FOOTREST press the switch under the armrest[1] or fold up the seat[2].The stairlift should beep whilst folding the footrest. Always ensure the footplate is in the down position during travel.

HINGED RAIL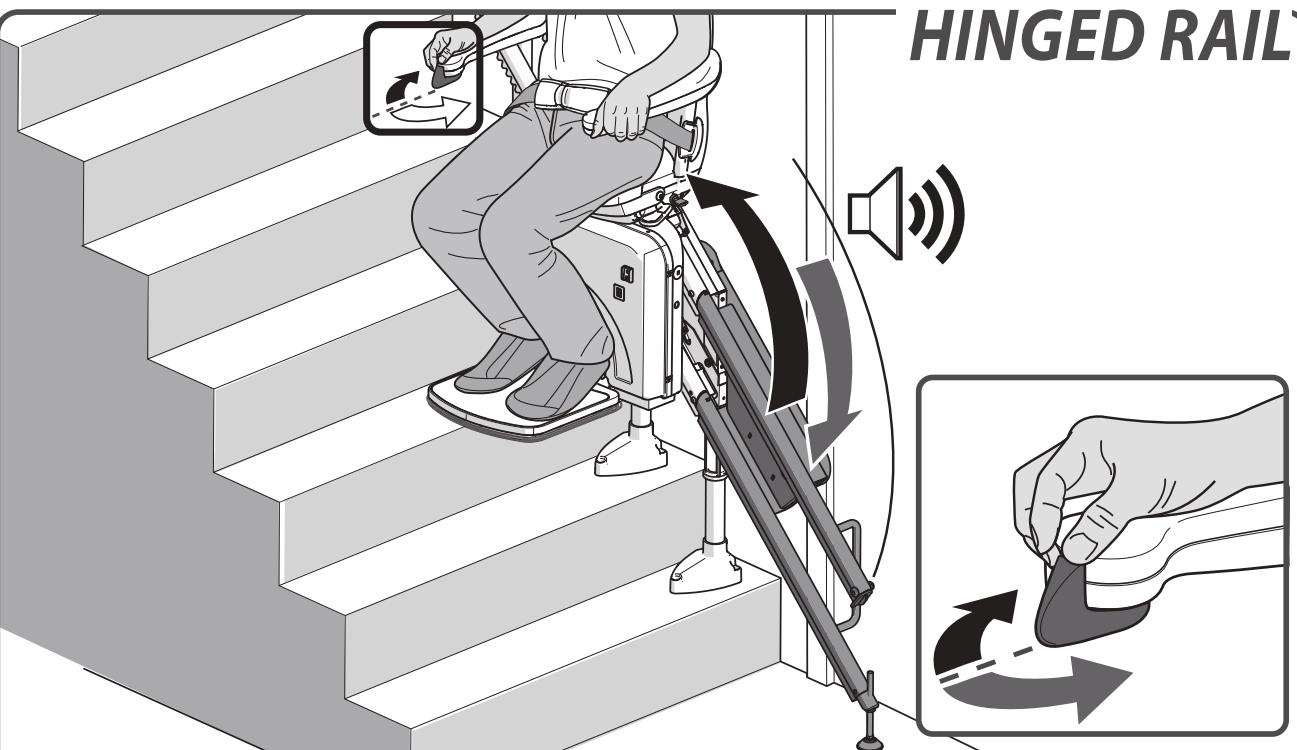


With the FOLDING HINGE option, the seat stops just above the hinged rail. Move the joystick up or down to fold or unfold the rail.

HINGED RAIL [CHARGING]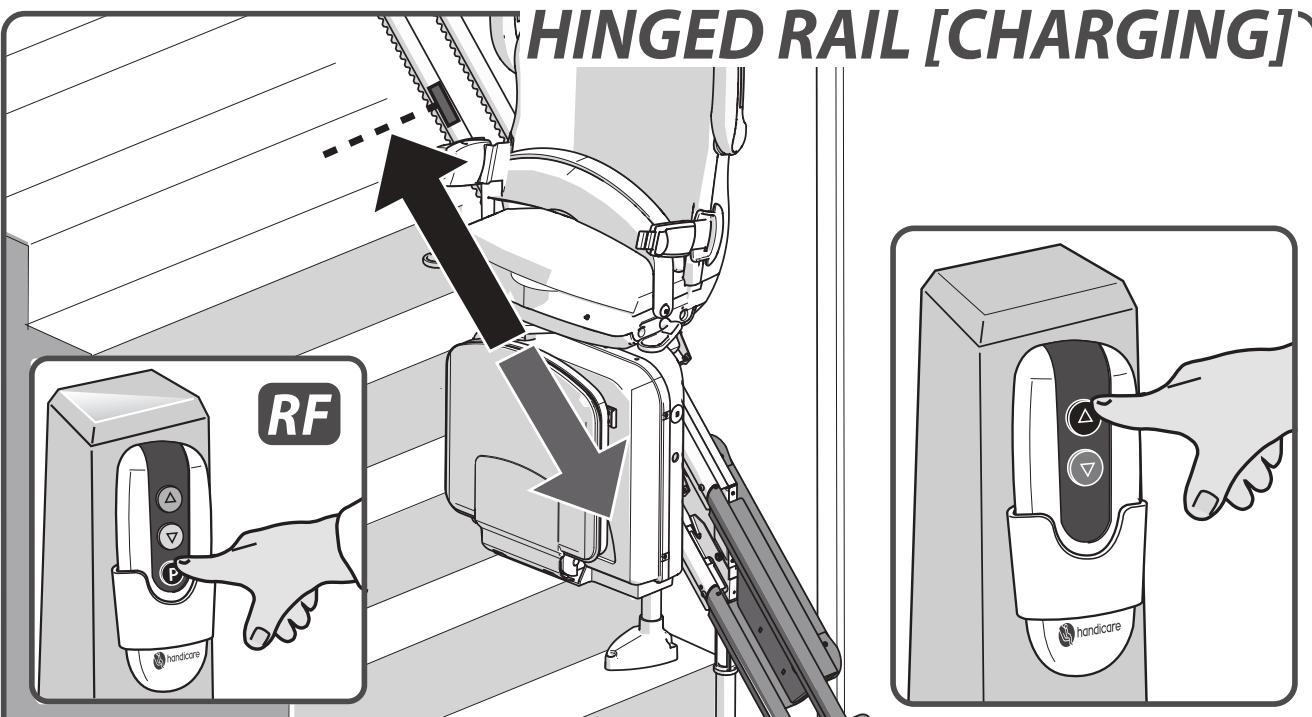

Do not leave the lift on the hinged rail after use. Use the remote control to move the lift to a charge point and recharge the batteries. The first stopping point is a charge point.

HINGE EMERGENCY OPERATION

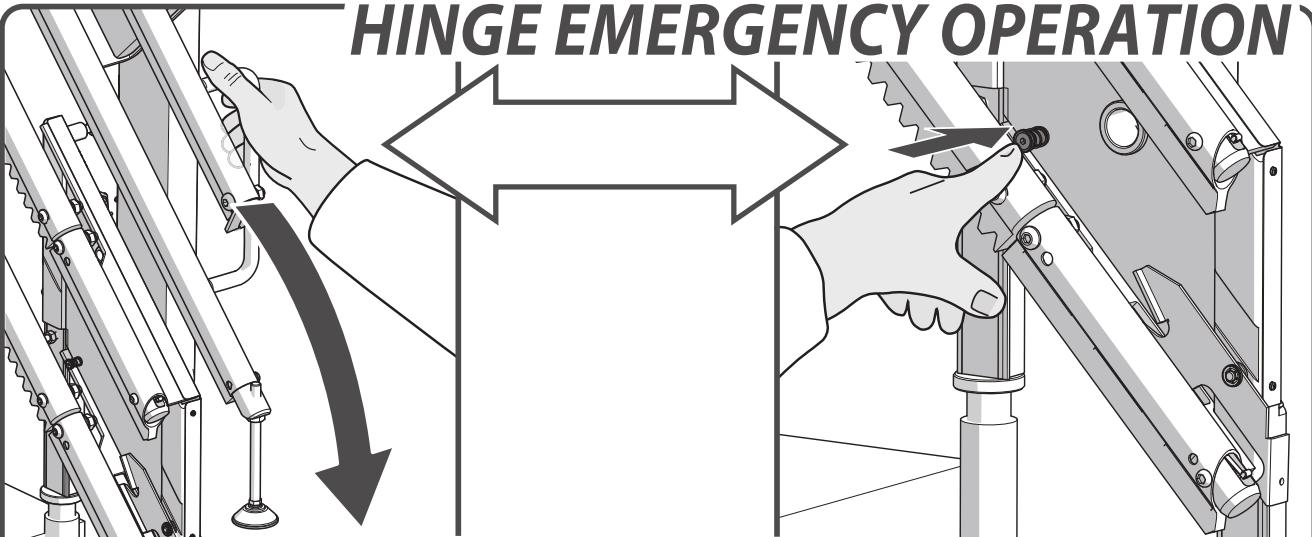If you are using the stairlift and the hinge system fails, please return to upper landing and call for assistance. Do not attempt to disembark mid-travel, only disembark at the upper or lower floors.

To fold the hinge manually In case of emergency: push the release button and move the main arm at the same time.

Tip: Loading of the main arm during normal operation can cause the pinion button to appear to be resistant to movement. Moving the main arm at the same time as pushing the release button will reduce the pressure on the pinion and make movement easier.

IMPORTANT INFORMATION

EC DECLARATION OF CONFORMITY FOR MACHINERY

(Directive 2006/42/EC)

Manufacturer: Handicare
 Address : 82 First Avenue
 Pensnett Estate
 Kingswinford,
 West Midlands DY6 7FJ
 United Kingdom

Herewith declares that the

Handicare 2000

is in conformity with the provisions of the Machinery Directive 2006/42/EC, Low Voltage Directive 2014/35/EU and EMC Directive 2014/30/EU.

The stairlifts are 33V DC Powered Stairlifts moving in an inclined plane, used with an approved battery charger product.

The stairlift has been designed and manufactured in accordance with the following standards:

EN 81-40, ISO 9386-2, EN ISO 12100-1,
 EN ISO 12100-2, EN ISO 14121,
 EN60204-1, (including the EMC standards).

Liftinstituut B.V., Buikslotermeerdijk 361, 1025 XE, Amsterdam, The Netherlands.
 Identificationnr 0400 has issued an EC-certificate in compliance with above mentioned directives and standards with reference NL13-400-1001-120-02

The Technical Construction File for the above Stairlift is retained at the manufacturers address. A copy is retained at the Liftinstituut B.V.office.

Kingswinford, United Kingdom, 19 july 2017

P. Slack
 Vice President, Operations, Handicare

Project:

Date:

FOREWORD

Your Handicare 2000 stairlift is a sophisticated product that has been specifically developed to carry one person (max 115 kg/18.1 stone/253 lbs) sitting on the seat (heavy duty type: max 137 kg/21.5 stone/301 lbs). Before using your Handicare 2000 stairlift please read this manual thoroughly to familiarise yourself with how to operate your stairlift.

Your stairlift has been constructed with state of the art technology and in accordance with recognised safety regulations so you can be sure that you have a safe and reliable solution.

Nevertheless, incorrect use of the stairlift may cause risk to the user or third parties, damage to your stairlift or other goods. Use your stairlift only when it is in good working order, in accordance with the intended use described above and with due attention to this manual. Always keep this manual near the stairlift.

TECHNICAL SUPPORT

Should issues arise that are not covered by this document please contact your supplier. Their Service department staff will always be pleased to help you. To help them identify your product please have the following details to hand when you call them:

- this manual
- your name, address, postcode, town and telephone number
- stairlift serial number which can be found on the label located on the power pack underneath the seat.

SAFETY INSTRUCTIONS

When using the stairlift always observe appropriate safety regulations regarding risk of electric shock and physical injury.

1. Provide an unobstructed environment by ensuring that the entire track is free of obstacles.
2. Take environmental influences into account. Do not expose the lift to damp or extreme sunlight. Make sure that the environment is well lit. In case of outdoor lift: always put the cover over the lift when the lift is not in use.
3. The stairlift should only be used by individuals who have received the appropriate guidance. On installation the supplier is required to demonstrate how to use this stairlift and the user is required to sign the declaration confirming that this instruction has been received.
4. Stairlifts should only be used for carrying one person: do not use your stairlift to carry pets, goods or more than one person at a time, even if the load is below the maximum user weight. Keep children and pets away from the stairlift.
5. Remove the key after use to prevent unauthorised use.
6. Keep limbs and clothing away from moving parts. Make sure that wide or loose clothing cannot become trapped between the chair and the stairs or rail.
7. Ensure a safe position during use. Always ensure a correct, stable position. Please sit as far back in the seat as you can and place your feet as close to the centre of the footplate as you are able to. Always use the safety belt and never stand on the stairlift.
8. Take care at all times. Concentrate on the operation of the stairlift as releasing the operating toggle will cause the lift to stop. To restart the lift wait 3 seconds and push and hold the toggle in the direction of travel desired until you reach the landing or hallway.
9. Check for damage. Before use check the lift for external damage. Report unusual noises, such as squeaking and creaking, to your supplier. To avoid accidents repairs and preventive maintenance should only be carried out by engineers qualified to do so.
10. The stairlift must not be used in the event of a fire or as a means of escape during a fire.

11. If the stairlift is not fitted with a communication device, we recommend that you always have one to hand.

12. Only leave the stairlift at the designated points at the top and bottom of the rail.

Structure

The strength and the suitability in general of wall, staircases, terraces, upper floors, banisters, plinths etc. which are to support or surround the installation have been verified and ensured by the customer, who therefore accepts all relative responsibilities. Neither Handicare nor its authorised dealers may be held responsible for the suitability of the structures which surround and support the installation.

MAINTENANCE

Cleaning

Clean the stairlift regularly with a damp cloth; avoid excessive use of water. Ensure the track is kept free of dirt and dust. Over time the track wheels may leave a mark on the rail.

Preventive maintenance

In addition to cleaning your stairlift, periodic preventive maintenance is necessary for continued long-term use.

Your stairlift must be serviced at least once a year by a qualified person. If your stairlift has more than two users we recommend a minimum of two services per year.

Please make an electrical socket available to provide local lighting if required during inspection and servicing. Lighting to a minimum value of 50 lux shall be provided at the boarding points whilst the stairlift is in use. Most light bulb packaging will detail the lux value of the light bulb.

GUARANTEE AND LIABILITY

Term of warranty – this warranty is valid for a period of 24 months from the date that the product ships from our factory. The purchase price for the product must be paid in full in order for the manufacturer to release parts under this warranty.

Coverage – this warranty applies to the repair or replacement, at the manufacturer's discretion, of parts that fail due to defective material or workmanship. The manufacturer may, at its discretion, provide factory reconditioned parts. This warranty is provided to the Authorised Handicare Dealer on behalf of the final purchaser of the product and is not transferable. The manufacturer's warranty does not cover labour charges for the removal, repair or replacement of warranty parts.

Damage caused by normal wear and tear, overloading or misuse and the replacement of batteries in the remote control is not covered by this guarantee.

We recommend that your stairlift is serviced annually by an appropriately qualified stairlift engineer.

Under no circumstances can the manufacturer accept responsibility for death, personal injury, damage to property, or incidental, unforeseen consequences and/or costs or damage resulting from the use of your stairlift.

The carrying out of alterations to the product is not permitted and doing so will invalidate the warranty. The manufacturer is not liable for damage resulting from such alterations.

The manufacturer accepts no liability for damage or injury caused by the non or inadequate observance of the safety regulations and instructions in this manual, or by negligence during the installation, use, maintenance and repair of this stairlift.

Installation and servicing should only be carried out by dealers or installers/service engineers authorised by the manufacturer in accordance with local regulations.

The wall mounted remote controls operate on a radio frequency; Handicare is unable to accept responsibility for system inoperability in the unlikely event of interference from signals external to this stairlift.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment OFF and ON, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

TECHNICAL SPECIFICATIONS

Supply voltage	230 VAC	Speed	0.15 m/s (max.)
Frequency	50/60 Hz	Ambient temperature	0 - 40°C
Current	1 A	Relative humidity	20 - 80%
Power	360 watt	Usage	15 rides per 24 hours
Fuses	30 A		10 rides continuous
Load	(max.) 115 kg	Noise	56 dB

The information provided in this manual is based on general specifications regarding the construction, material properties and working methods known to us at the time of publication; alterations and improvements are therefore anticipated.

The configuration of the Handicare stairlift, including in particular (but not limited to) its parts such as the rail system, the construction of the stairlift, the fixation parts, all swivelling and movable parts, and the motor, rely on Handicare's know-how. Work on this configuration or on one or more of its parts may only be performed by dealers (and their employees) who have been certified by Handicare to perform such work in accordance with the instructions given by Handicare. Only these employees have received the required training; only these qualified employees can provide the level of safety that Handicare stairlifts have when delivered ex works. Suppliers of used Handicare stairlifts other than dealers who have been certified by Handicare are not qualified to perform such work, e.g. installing a rail for a user other than the original Handicare rail.


This is important, because when work is performed on a Handicare stairlift or a used Handicare stairlift is re-installed with other parts, such as a different rail, by anyone other than the aforementioned qualified employees, Handicare's warranty and the CE mark for the stairlift in question will become invalid with immediate effect.

Once the CE mark has become invalid, this means that the stairlift in question no longer complies with European regulations. As a result, Handicare will no longer be responsible or liable for any consequences arising from the failure of the stairlift in question. Furthermore, maintenance on the Handicare stairlift may only be performed using original spare parts that have been approved by Handicare.

This is important, because if parts other than original spare parts approved by Handicare, such as a different rail, are used, the Handicare warranty and the CE mark for the stairlift in question will become invalid with immediate effect.


All rights reserved. No part of this publication may be reproduced and/or published by means of print, photocopy, microfilm or in any other way or form, without prior written permission from the manufacturer. This also applies for the accompanying drawings and diagrams. The manufacturer reserves the right to change parts at any time, without prior or direct notification to the customer. The contents of this manual may also be changed without prior warning. This manual is intended for the standard models of the lift. The manufacturer therefore cannot be held liable for any damage resulting from specifications that differ from those of the standard model of the lift supplied to you. For information concerning adjustment, maintenance or repairs not provided in this manual, please contact your supplier's service department. This manual has been compiled with the best intentions and every possible care but the manufacturer cannot accept responsibility for any errors in this manual or for the consequences thereof. The Dutch manual is the original language.

DIAGNOSTIC DISPLAY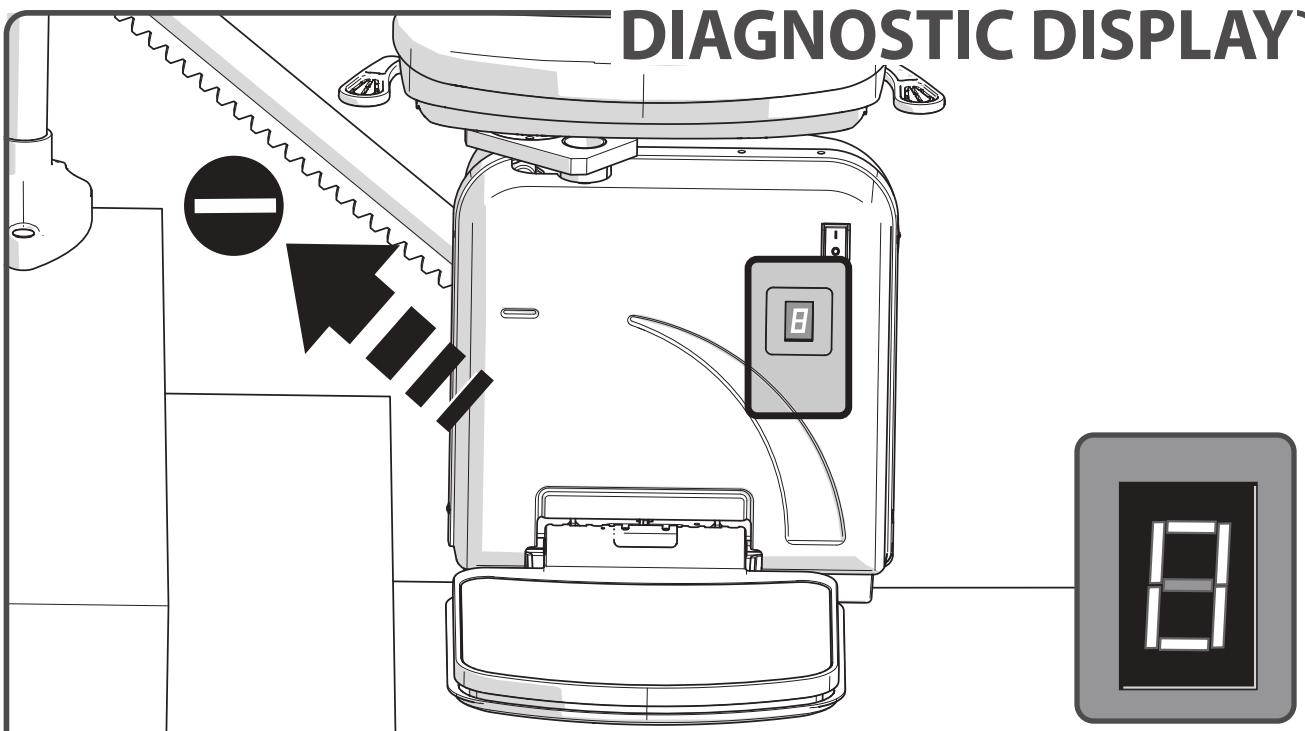

The stairlift is fitted with a diagnostic display that displays a code to help diagnose faults. Refer to APPENDIX E for an explanation of the diagnostic codes. Please inform the customer service operator of this code when reporting any fault.

CHAIR POSITION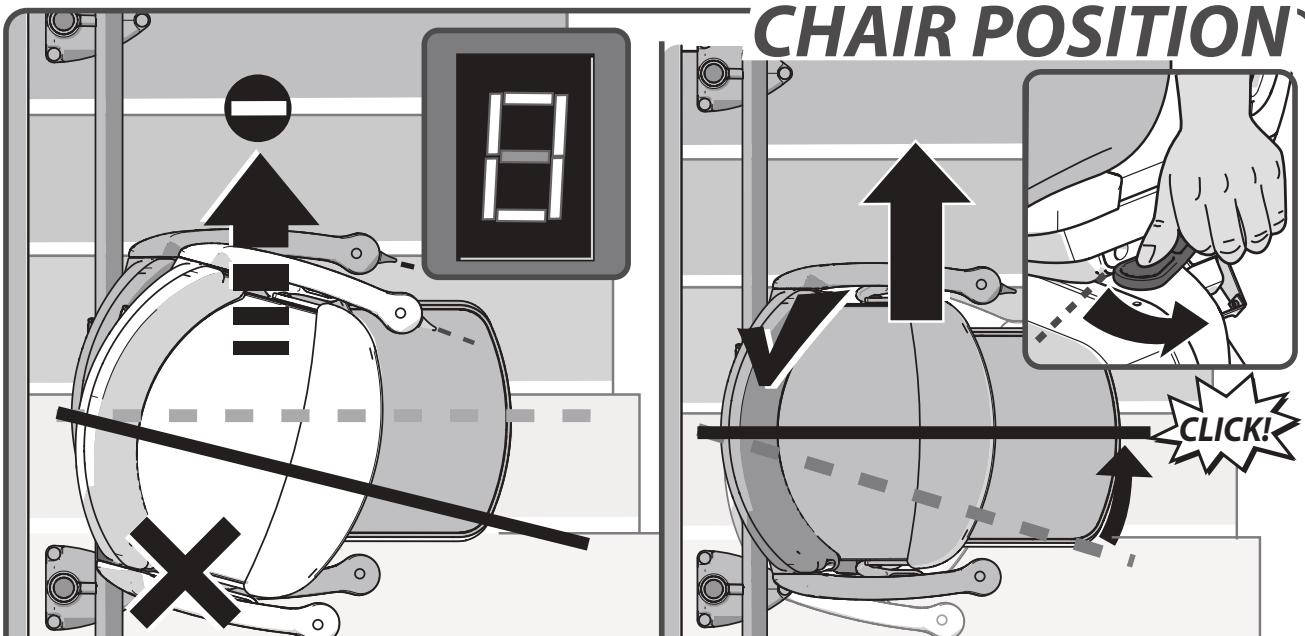

Problem: the lift does not travel in either direction and the display shows a "0".

Cause: the seat is not locked and is in a swivelled position.

Solution: swivel the seat using the handle located to the left or right under the seat, until it "clicks" into position.

STOP BUTTON

CAUSE

SOLUTION

Problem: the lift does not travel in either direction, the display shows a “0” [1] and the stop button [2] is illuminated.

Cause: the stop button has been pressed.

Solution: press the stop button once.

OPTIONAL STOP BUTTON


CAUSE

SOLUTION

Problem: the lift does not travel in either direction, the display shows a “0” [1] and the stop button [2] is illuminated.

Cause: the stop button has been pressed.


Solution: turn the emergency stop button.

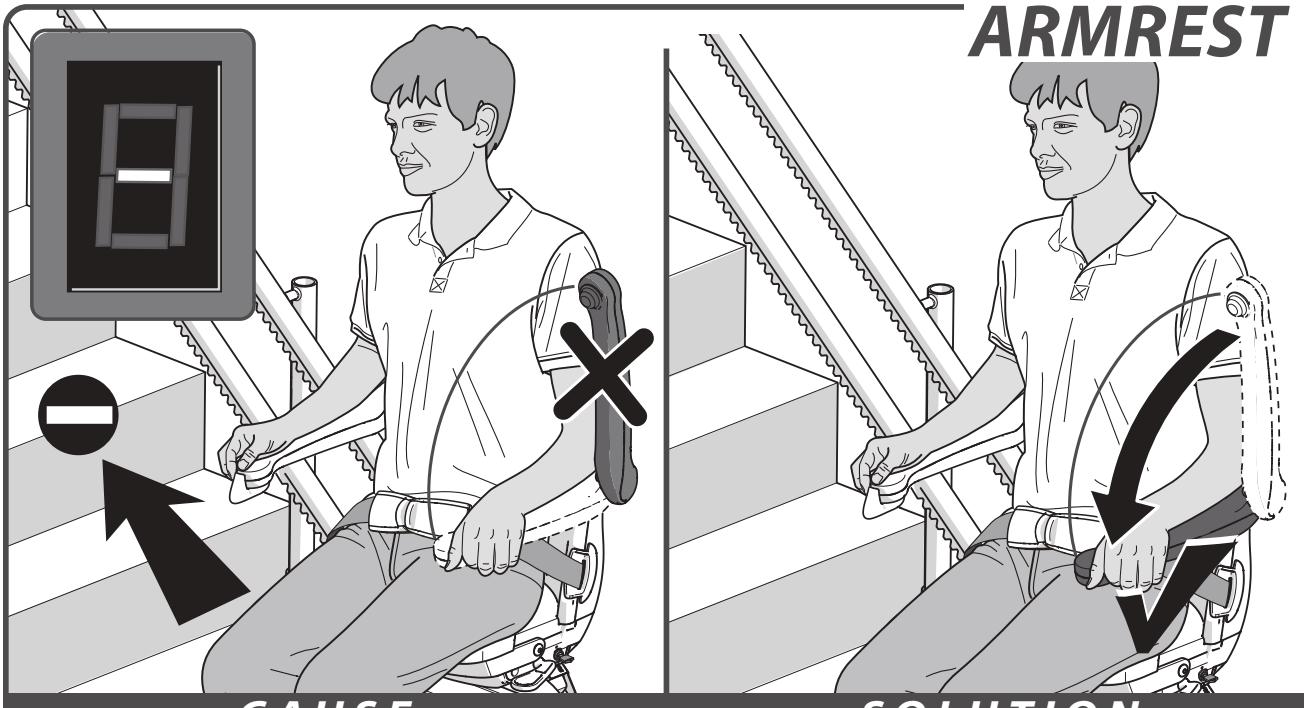**CAUSE****SOLUTION**

Problem: the lift does not travel in either direction and the display shows a “-”.

Cause: an armrest has not been folded down.

Solution: fold down the armrest fully.

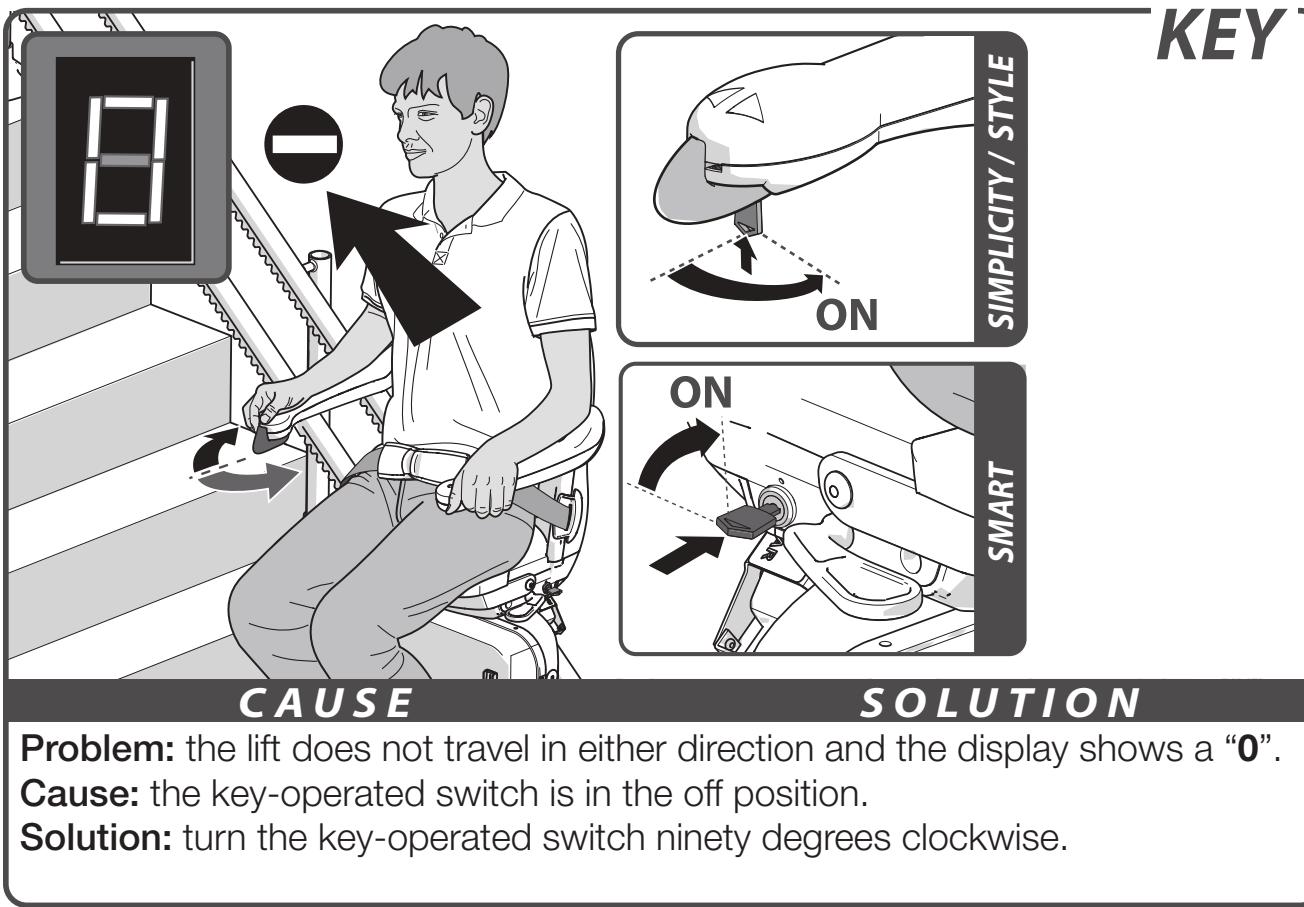**CAUSE****SOLUTION**

Problem: the lift does not travel in either direction and the display shows a “0”.

Cause: the key-operated switch is in the off position.

Solution: turn the key-operated switch ninety degrees clockwise.

BATTERIES RC

CAUSE

SOLUTION

Problem: the lift does not travel in either direction and the LED on the remote control does not light up when the remote control is operated.

Cause: the batteries in the remote control are flat.

Solution: replace the batteries in the remote control.

POWER



CAUSE

SOLUTION

Problem: the lift batteries are not recharging, the lift beeps for 15 seconds and the display shows a “2”.

Cause: the lift is not in the correct position at a charge point or the plug is not in the socket.

Solution: move the lift to a charge point or put the plug in the socket.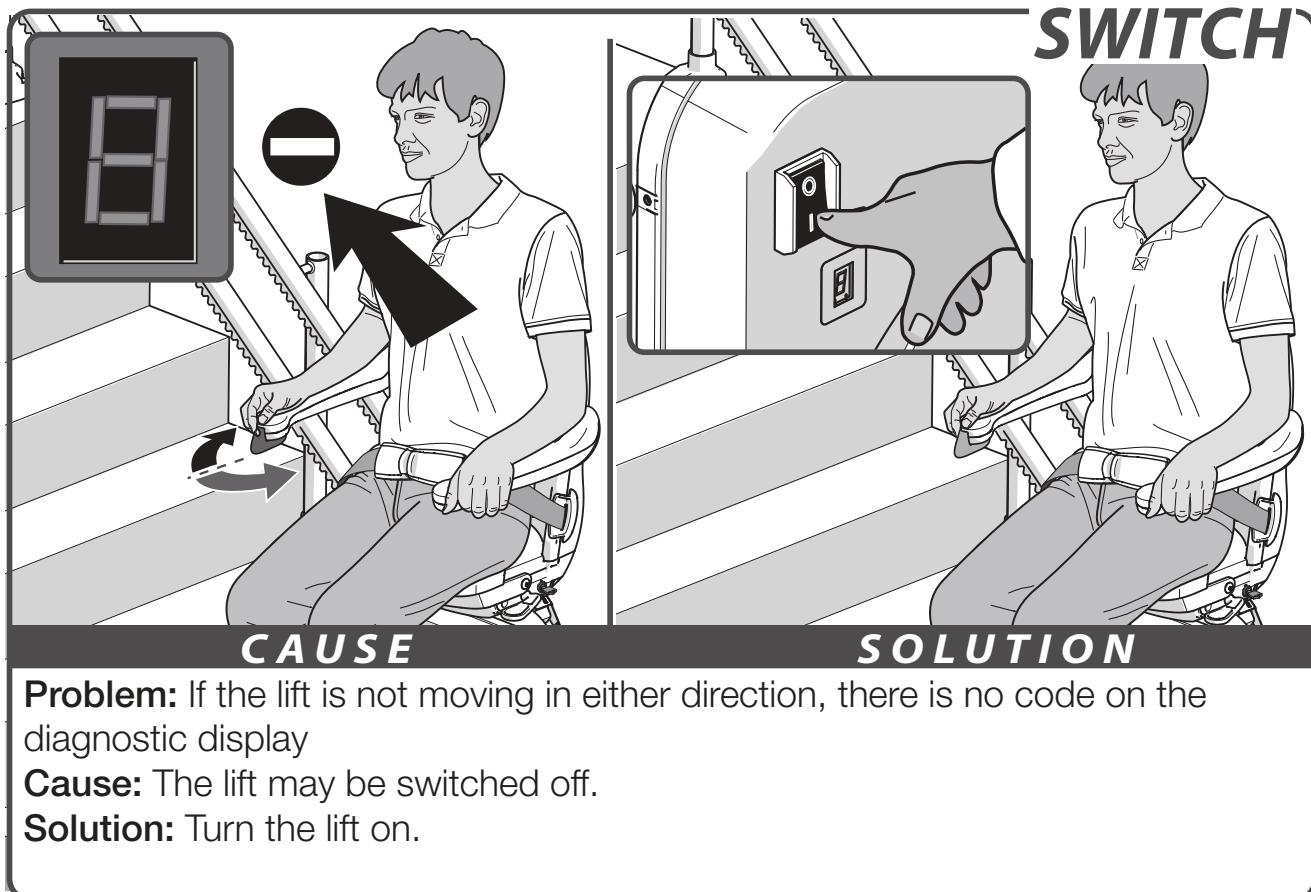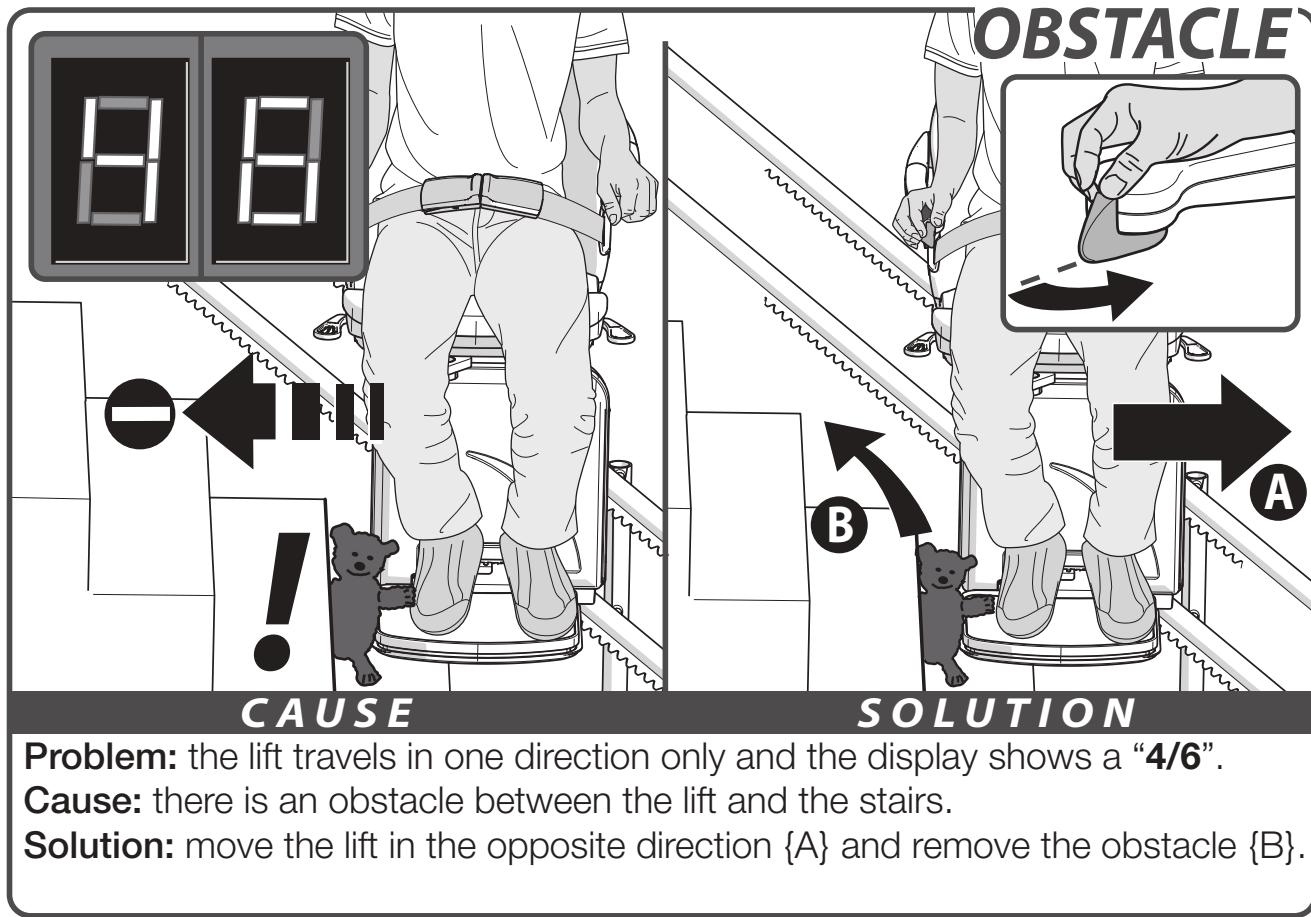


ATTACHMENTS


REMINDERS _____ page 23

EMERGENCY OPERATIONS _____ page 24

DIAGNOSTIC CODES _____ page 27

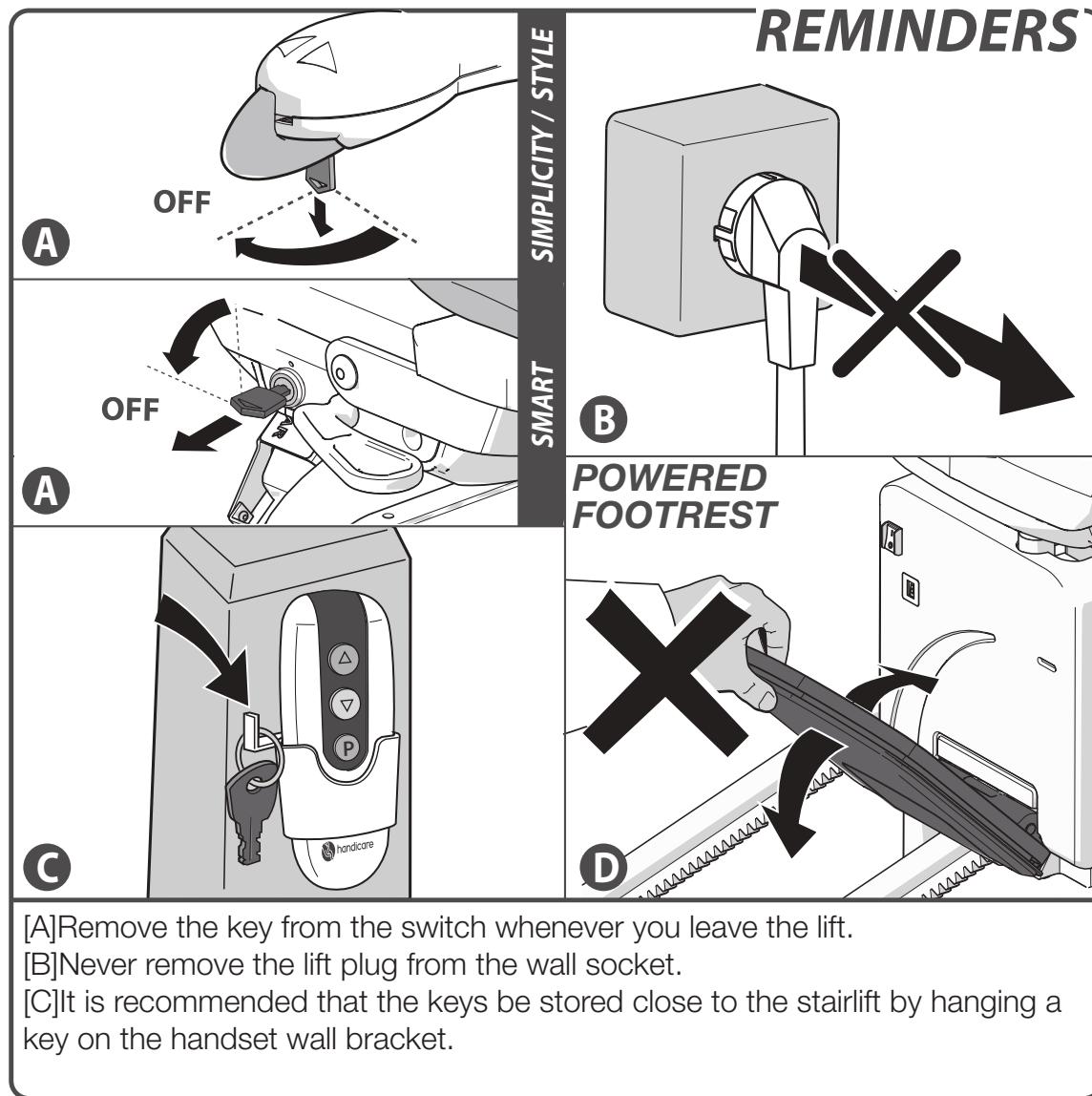


EMERGENCY OPERATION > TRACK




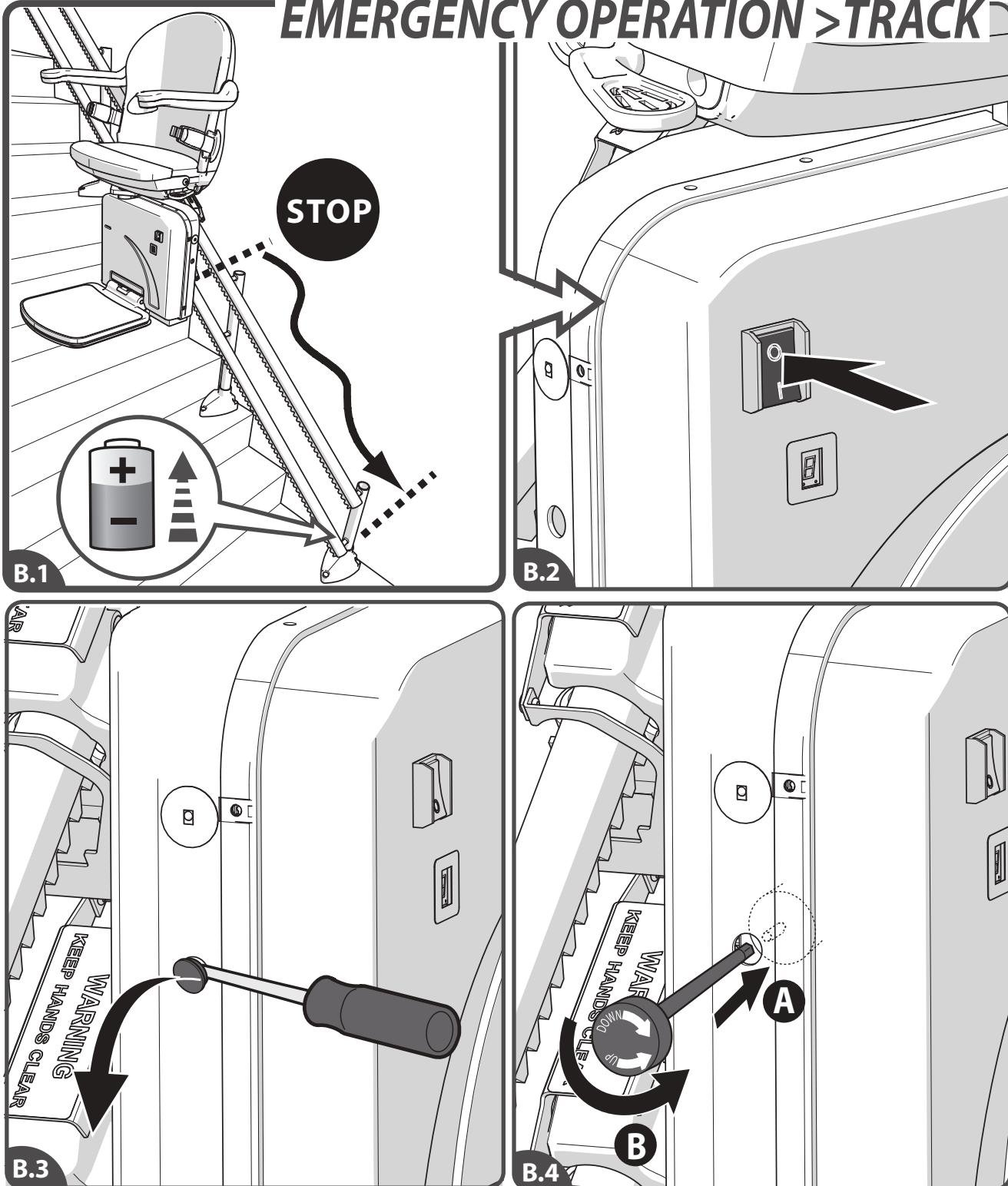
[B.1] When the stairlift stops unexpectedly and no longer moves any further, the hand winder can be used to move the lift to a charging point. [B.2] Turn off the lift. [B.3] Remove the cap from the unit with a screwdriver. [B.4] Turn the hand winder as indicated on the winder. Then wind the lift in the required direction. If it is not possible to wind the lift in the downward direction, wind the lift upwards. The lift must be serviced before it can be used again.

EMERGENCY OPERATION > SMART SEAT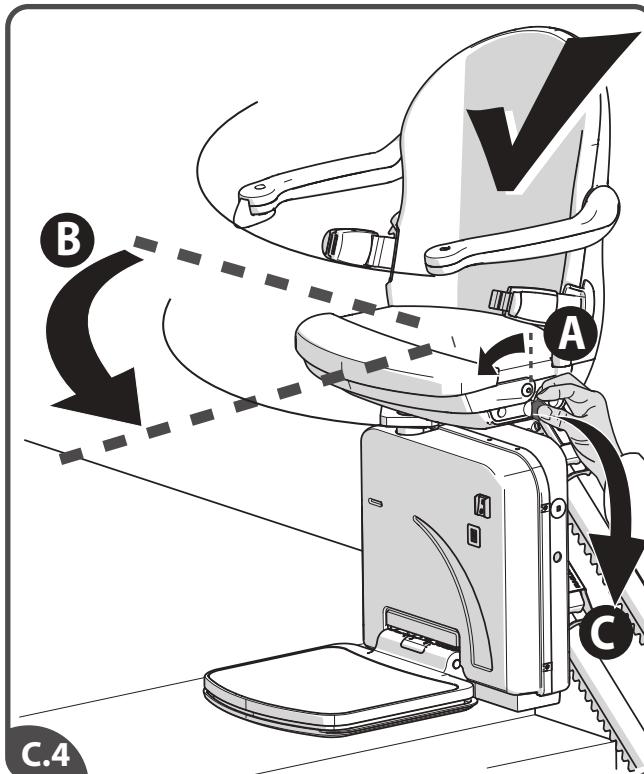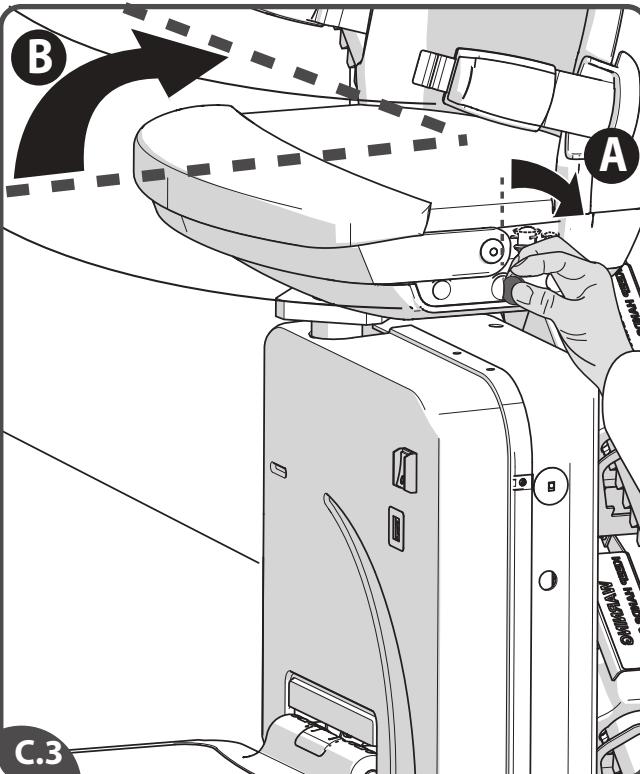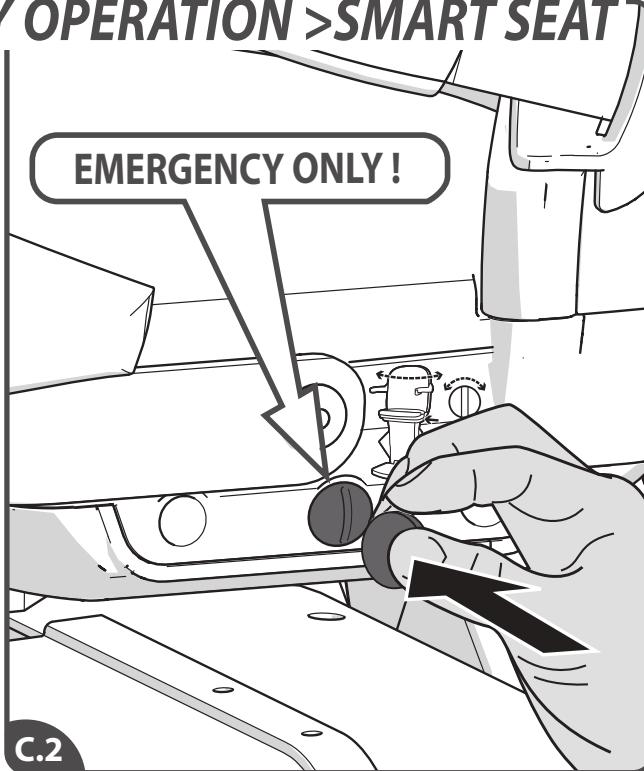



[C.1] If the power swivel is not functioning it can be swivelled manually. Do this only in an emergency. [C.2] Insert a coin in the button on the side of the seat. [C.3] Swivel the seat manually. [C.4] The seat must be returned to the start position. Remove the coin. Do this before you use the lift again.

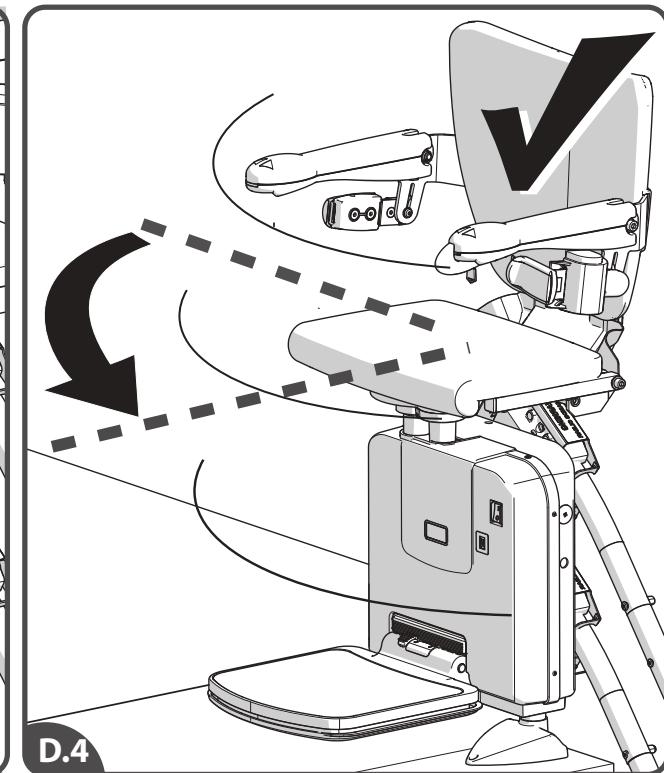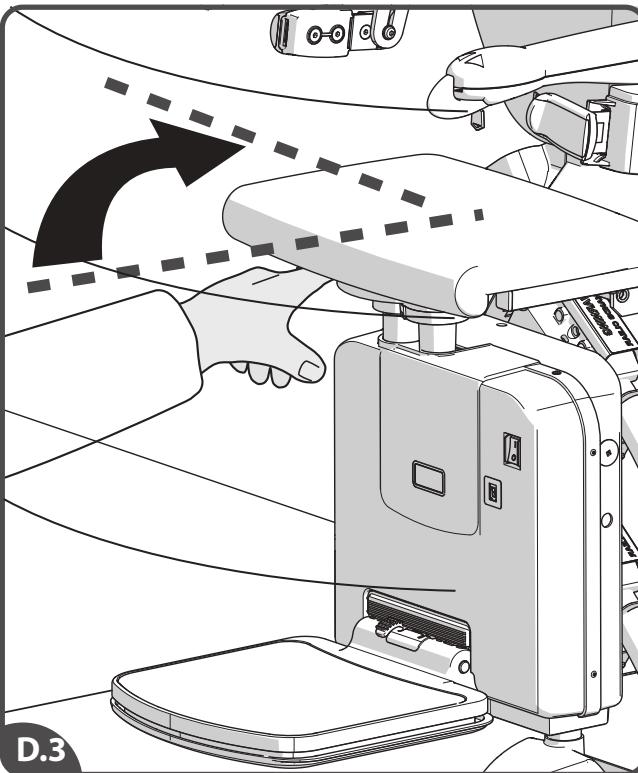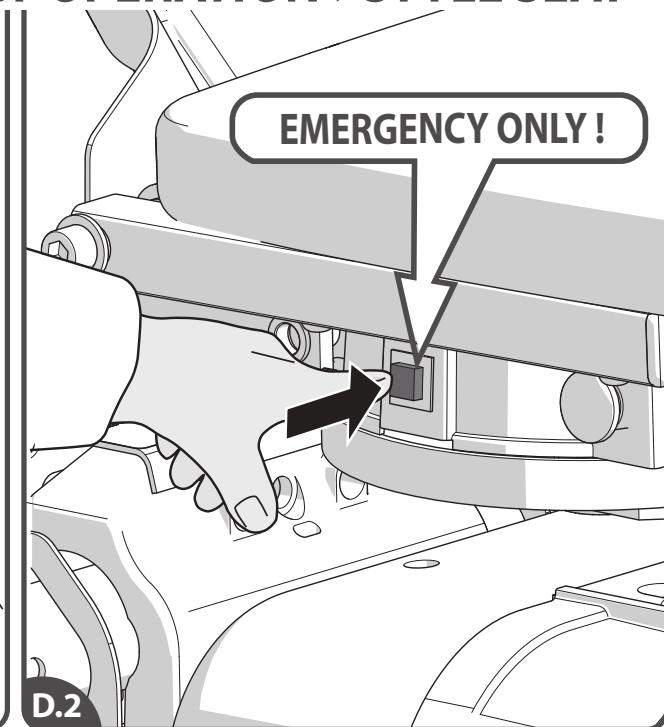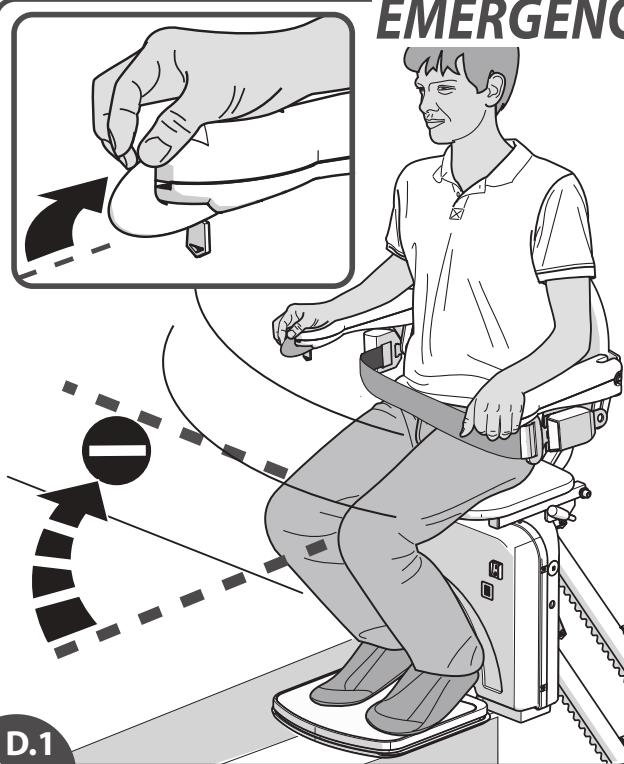EMERGENCY OPERATION >STYLE SEAT









[D.1]If the power swivel is not functioning it can be swivelled manually. Do this only in an emergency. [D.2]Press the release button. [D.3]Swivel the seat manually. [D.4]The seat must be returned to the start position. Do this before you use the lift again.

If the mentioned action does not help, call Handicare.




DIAGNOSTIC CODES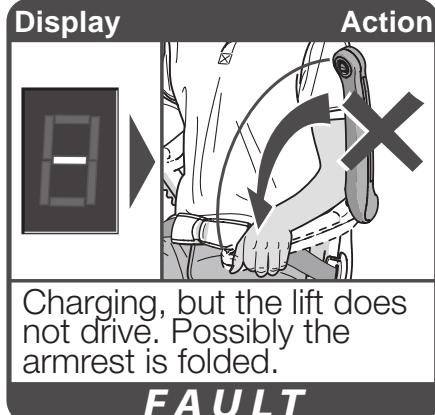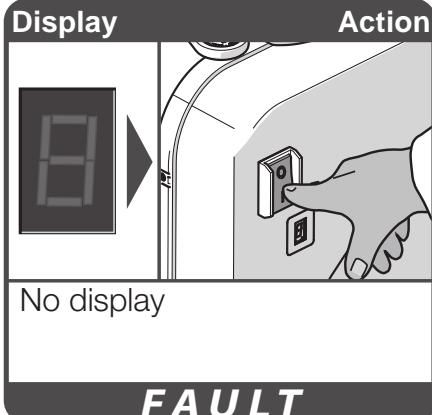

Write down the diagnostic code when you call Handicare for help.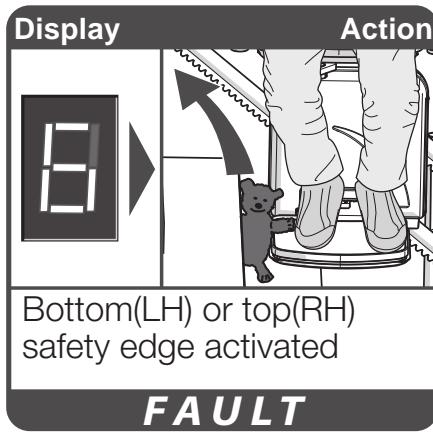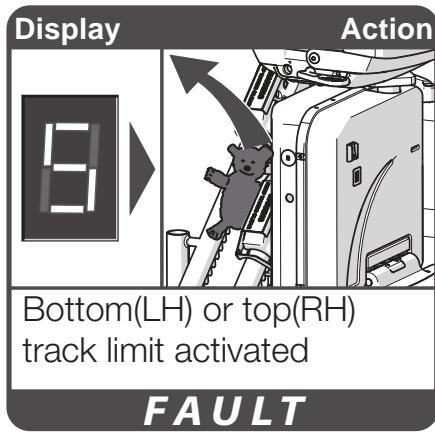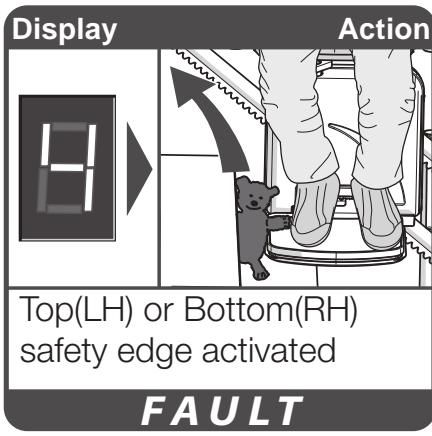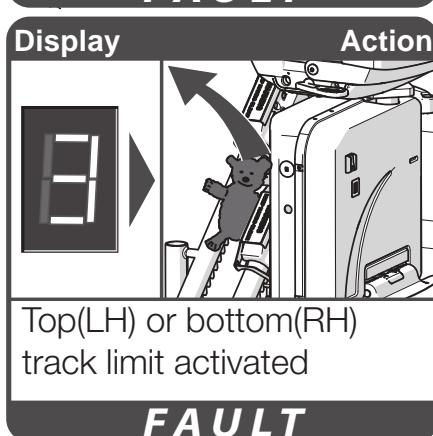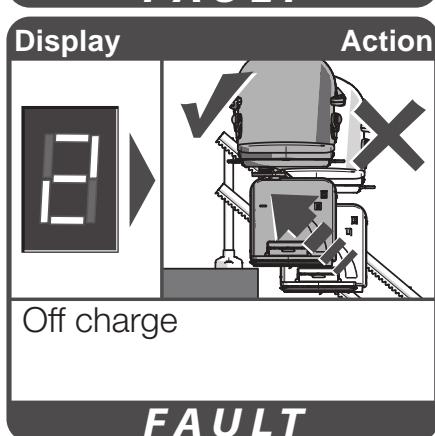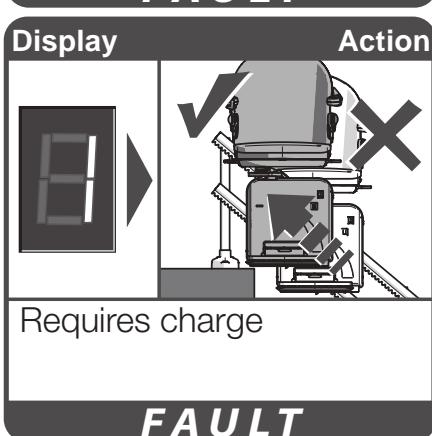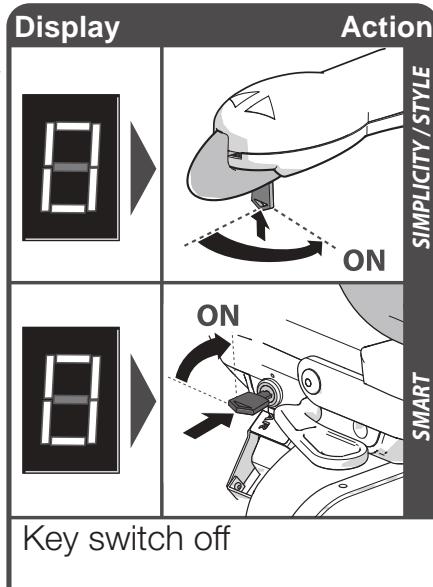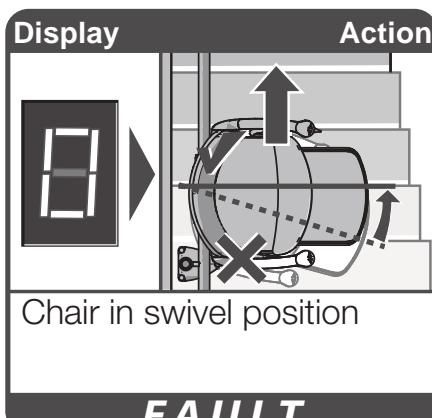

Display	Action
Low battery voltage	
FAULT	
Display	Action
Joystick activated	
FAULT	
Display	Action
Joystick activated	
FAULT	
Display	Action
IR address fail	
FAULT	
Display	Action
The main power relay not open	
FAULT	
Display	Action
The main power not closed	
FAULT	
Display	Action
Brake semi-conductor failed	
FAULT	
Display	Action
Brake not connected	
FAULT	
Display	Action
The main power relay has closed early	
FAULT	
Display	Action
Current limit exceeded	
FAULT	
Display	Action
Lift driving at half speed	
FAULT	
Display	Action
The main control board has a memory problem	
FAULT	

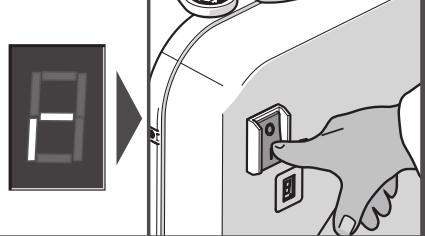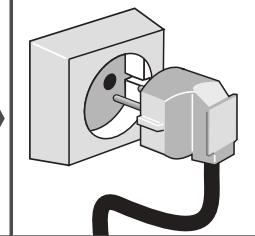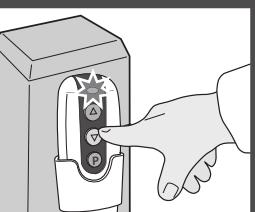Display		Action
		
Power supply fault during charging		
FAULT		
Display		Action
		
Power supply fault during charging		
FAULT		
Display		Action
		
Power supply fault when battery on float		
FAULT		
Display		Action
		
Main board hardware fault		
FAULT		
Display		Action
		
Over current		
FAULT		
Display		Action
		
Sleeping mode		
FAULT		
Display		Action
		
Sleeping mode		
FAULT		
<p>If the display is showing anything but the diagnostic codes listed it requires resetting: drive the lift off the charge contacts. Turn the main switch on the unit off for 10 seconds and then back on.</p>		

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PILES

FR

The diagram is divided into two sections: 'CAUSE' on the left and 'SOLUTION' on the right.

CAUSE: A hand holds a remote control with a red light indicator. An inset shows a battery with a minus sign at the top and a plus sign at the bottom, followed by an exclamation mark, indicating low power.

SOLUTION: A hand is shown removing two batteries from the remote control's compartment. Another hand is shown inserting new batteries into the compartment, with arrows indicating the correct orientation (+ and - terminals).

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ALIMENTATION

The diagram is divided into two sections: 'CAUSE' on the left and 'SOLUTION' on the right.

CAUSE: A hand holds a power adapter connected to a wall outlet. Below it, a digital timer shows '0' and '15 sec.' with a speaker icon, indicating a 15-second bip. To the right, a robotic lift chair is shown with a large 'X' over its backrest, indicating it is not functioning. An inset shows a battery with a minus sign at the top and a plus sign at the bottom, with a downward arrow, indicating low power.

SOLUTION: A hand is shown plugging the power adapter into a wall outlet. The robotic lift chair is shown with a checkmark over its backrest, indicating it is now functioning. An inset shows a battery with a plus sign at the top and a minus sign at the bottom, with an upward arrow, indicating a full charge.

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BATTERIE TC

IT

The diagram is divided into two sections: 'CAUSA' (Cause) on the left and 'SOLUZIONE' (Solution) on the right.

CAUSA: A hand holds a remote control with a red LED light labeled 'RED light'. An inset shows a battery with a minus sign and an exclamation mark, indicating low power.

SOLUZIONE: A hand is shown removing two batteries from the remote control. Another hand is shown inserting new batteries into the remote control.

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The diagram is divided into two sections: 'CAUSA' (Cause) on the left and 'SOLUZIONE' (Solution) on the right.

CAUSA: A display shows a large 'E' (empty). Below it, a timer indicates '15 sec.' and a speaker icon. An inset shows a battery with a minus sign and a downward arrow, indicating low power.

SOLUZIONE: A hand is shown plugging a power cord into a wall outlet. The lift chair is shown with a checkmark indicating it is correctly connected to the power source.

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Handicare GmbH
Handicare USA, Inc.