




EvaDrive™

# QuickTips


Weight Capacity:  
EvaDrive600- 600lb SWL  
EvaDrive450- 450lb SWL

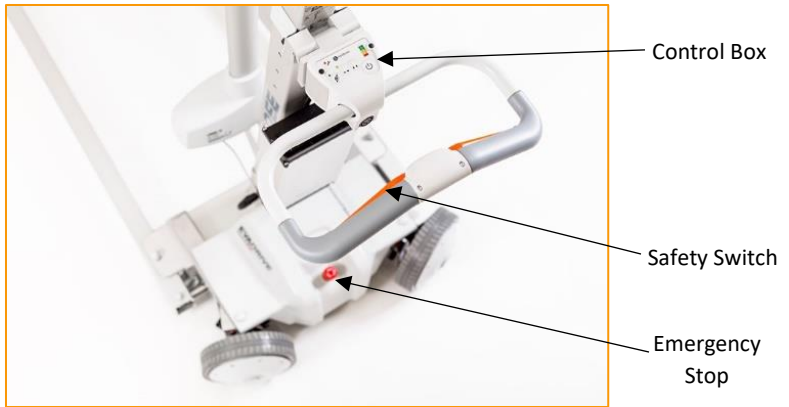

Control Box  
Safety Switch  
Emergency Stop



Safety Switch

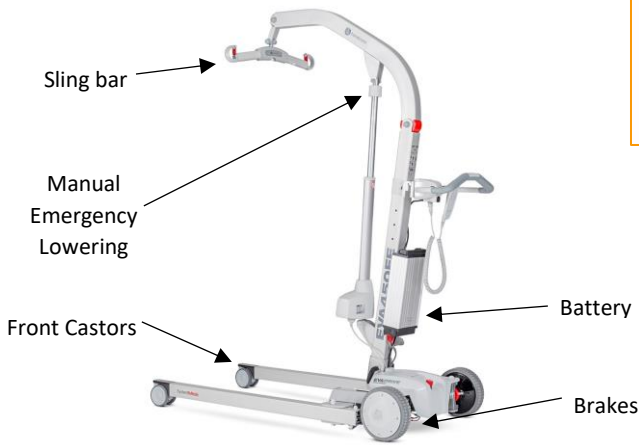

Sling bar  
Manual Emergency Lowering  
Front Castors  
Battery  
Brakes

- To “Drive” the **EvaDrive**, turn the device on with the **Power Button**, wait at least 2 seconds.
- Ensure proper battery charge. Indicate the direction of travel (forward/backward, lateral, or rotation) by using the rear wheel direction controls, turn the wheels in the desired direction.
- Engage the **Safety Switch** as seen here and apply slight pressure forward/backward to begin the “Drive” operations. The speed will increase/decrease as you walk faster/slower.
- To operate lift for patient transfer, use desired sling, sling bar, and hand control in the typical manner; similar to the non-drive enabled lift. Once patient is safely secured, use drive feature during transfer to reduce push/pull force risk.
- For complete instructions see the official *EvaDrive User Manual*.



Red indicator light: Service required  
Yellow indicator light: Wheel turning activated  
Green indicator light: Low Speed Activated  
Low Speed Button  
Rear wheel direction controls  
Battery Level Indicator  
Power Button

**NOTE: When returning wheels to forward direction, hold forward direction on directional controls until the yellow light turns off**